



Flight 60 SW 4.31 Version Release

- R&D QA GN
- PR CS

COVER PAGE

APPROVAL				
	Name	Title	Date	Signature
Prepared by	Talia Sirkis	Product Manager	15.10.2018	Signed
Reviewed by	Ken Raichman	QA&RA manager	15.10.2018	Signed
Approved by	Amir Kleinstern	CEO	15.10.2018	Signed

REVISION HISTORY		
Date	Revision #	Change Description
15.10.2018	A00	4.31 SW version release

Flight 60 SW Version Release


Version 4.31

New features:

1. Improvements of External DC performances

2. New Ventilation state – Stand by

The user no longer goes directly to shutting down the ventilator when stopping ventilation.

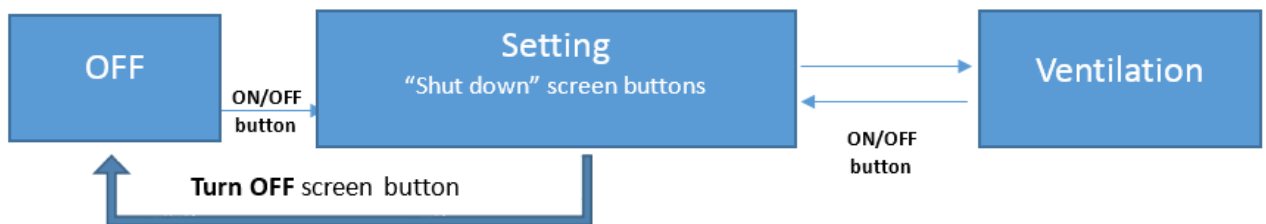

Known Issues

- Updating from version 4.21/4.30 to 4.31, cause a **fault** “Customize BUV is ON” caution. Pressing alarm reset hides the caution.
- APNEA interval does not back to default value when the custom BUV is disabled (password protected)