



## Flight 60 SW 4.30 Version Release note

R&D    QA    GN  
 PR    CS

---

---

### COVER PAGE

---

---

APPROVAL				
	Name	Title	Date	Signature
Prepared by	Shimrit Weinberg	Product manager	14.10.21	Signed
Reviewed by	Vitaly Dubrovner	R&D SW manager	14.10.21	Signed
Reviewed by	Ken Raichman	QA & Regulatory manager	14.10.21	Signed
Approved by	Amir Kleinstern	CEO	14.10.21	Signed

# *Flight 60 Turbine SW Version Release*

## *Version 5.34*

### SW changes:

#### **1. Bug fixing**

- 1.1. In configuration without internal oxygen mixer, when choosing the "Low MV" alarm button on the main screen, it was not responding, and the corresponding value could not be updated by user. This bug is fixed.
- 1.2. Apnea PC value could be set without limitation by PEEP, so the total target pressure could be over 80 cmH<sub>2</sub>O. This bug is fixed.
- 1.3. On the service screen the window for updating "Total vent hours" appeared on the wrong place of the display. This bug is fixed.
- 1.4. When the "Low Rate" alarm is ON, "Low Rate" alarm appears during the "Circuit Test. This bug is fixed.

#### **2. Translations**

- 2.1. Updating Polish translations.