



- R&D
- QA
- GN
- PR
- CS

COVER PAGE

APPROVAL				
	Name	Title	Date	Signature
Prepared by	Shimrit Weinberg	Product manager	26.10.22	Signed
Reviewed by	Vitaly Dubrovner	R&D SW manager	26.10.22	Signed
Reviewed by	Ken Raichman	QA & Regulatory manager	26.10.22	Signed
Approved by	Amir Kleinstern	CEO	26.10.22	Signed

Flight 60 Turbine SW Version Release

Version 5.35

This document is the release notes for limited SW version release #5.35 for Flight-60 Turbine ventilator.

1 CHANGES TO RELEASE

1.1 SW changes:

1.1.1 Bug fixing

- JSON messages (Bug #6059)
- Problem found in the previous version: JSON messages are corrupting when ventilation is starting
- Cause: debug information sent together with JSON messages
- Solution: remove sending debug information during the ventilation

1.1.2 New Features

No new features added

2 OPERATING SYSTEM SUPPORTED

Flight 60T – Turbine all versions

3 SYSTEM REQUIREMENTS

NA

4 NEW OR DROPPED FEATURES

No new or dropped features

5 KNOWN ISSUES AND LIMITATIONS

The following known issues and limitations exist for SW Release 5.35:

- Only minor problems: "Apnea Backup parameters" are not translated into other languages
- Some of the translations are out of the frame (in Polish and Russian)
- "Low Rate" alarm may appear during "Circuit test" if the limit is high (≥ 20)

6 FILES ISSUED


fm02132.pkg

7 DEPLOYED INSTRUCTIONS

7.1 Software Upgrade required equipment:

USB storage device with the SW version (**fm02132.pkg**)

7.2 Installing the SW upgrade

- Verify the batteries charge percentages are above 50%.
- Turn off the ventilator, disconnect from AC.
- Copy the new SW version (**fm02132.pkg**) to a USB storage device.
- Plug the USB storage device to the vent USB port.
- Connect the ventilator to AC.
- The ventilator powers up and a software update screen opens, displaying the current version and the new version to update.
- Press OK on the touch screen.
- Once upgrade is done, the ventilator displays " Finished updating, please remove DOK"
- Remove the USB storage device and press **OK** on the touch screen.
- Turn on the ventilator, press **Technical** /  and press **Show Info**.
- Verify the new software version is displaying.

8 RELATED DOCUMENTATION

- Operating Manual - LIT-0089 B07
- Service Manual - LIT-0113 B08