



Flight 60 SW 4.35 Release Note

- R&D QA General
 Production

COVER PAGE

APPROVAL				
	Name	Title	Date	Signature
Written by	Shimrit Weinberg	Product manager		
Reviewed by	Yarden Meshulam	SW manager		
Reviewed by	Ken Raichman	QA & Regulatory manager		
Approved by	Amir Kleinstern	CEO		

REVISION HISTORY		
Date	Revision #	Change Description
	A00	Initial version

R&D QA General
 Production

CONTENTS

<u>1. NEW OR DROPPED FEATURES</u>	<u>3</u>
<u>2. CHANGES TO RELEASE</u>	<u>3</u>
2.1. BUG FIXES	3
<u>3. KNOWN ANOMALIES</u>	<u>3</u>
<u>4. OPERATING SYSTEM SUPPORTED</u>	<u>3</u>
<u>5. SYSTEM REQUIREMENTS</u>	<u>3</u>
<u>6. FILES ISSUED</u>	<u>3</u>
<u>7. DEPLOYED INSTRUCTIONS</u>	<u>3</u>
7.1. REQUIRED EQUIPMENT	3
7.2. PROCESS	4
<u>8. RELATED DOCUMENTATION</u>	<u>4</u>

- R&D QA General
 Production

1. NEW OR DROPPED FEATURES

- There are no new or dropped features in SW version 4.35

2. CHANGES TO RELEASE

2.1. Bug fixes

- A bug causing the wrong value to be sent through the RS232 port when using Jason format was fixed
- A bug in the UI translation to Russian & Spanish was fixed

3. KNOWN ANOMALIES

- ALARMS log may be full with batteries fault alarms on and off on every breath if the batteries are close to be fully discharged.
- This release version is not fully translated to other languages. There will be an additional release version after translating
- There may be “artifacts” on the screen after changing languages
- There may be “artifacts” on the screen after enabling/disabling presets on the service screen

4. OPERATING SYSTEM SUPPORTED

All of Flight 60 piston ventilator versions.

5. SYSTEM REQUIREMENTS

NA

6. FILES ISSUED

- fm02144.pkg

7. DEPLOYED INSTRUCTIONS

7.1. Required equipment

- USB storage device with the new SW version (*.pkg* file)

- R&D QA General
 Production

7.2. Process

- Turn off the F60 properly
- Remove all power sources (M.battery, S.Battery, AC)
- Insert the USB mention in step 7.1
- Reconnect the all the power sources
- Turn on the device
- Verify that you see on the screen the latest version under the field “SW version”
- Press “Start”
- At the end of the process, the ventilator will shut down.

8. RELATED DOCUMENTATION

- LIT-0014 Rev A15 F60 Operating Manual
- LIT-0021 Rev A14 Flight 60 service manual