

 Flight Medical®	Ventoux SW 1.25 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1028

COVER PAGE

APPROVAL				
	Name	Title	Date	Signature
Written by	Shimrit Weinberg	Product manager	Per Arena	Per Arena
Updated by	Yarden Meshulam	R&D SW manager	Per Arena	Per Arena
Reviewed by	Eyal Ozeri	QA & Regulatory manager	Per Arena	Per Arena
Approved by	Amir Kleinstern	CEO	Per Arena	Per Arena

 Flight Medical®	Ventoux SW 1.25 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1028

CONTENTS

1. GENERAL 3

2. NEW OR DROPPED FEATURES 3

2.1. DROPPED FEATURES 3

2.2. NEW FEATURES..... 3

3. CHANGES TO RELEASE 3

.3.1 BUG FIX..... 3

4. KNOWN ANOMALIES 3

5. OPERATING SYSTEM SUPPORTED 3

6. SYSTEM REQUIREMENTS 4

7. FILES ISSUED 4

8. DEPLOYED INSTRUCTIONS 4

9. RELATED DOCUMENTATION 4

	Ventoux SW 1.25 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1028

1. GENERAL

SW update to version 1.25 is recommended.

For a full coverage of all bugs is it also recommended to upgraded MB to version A05 (please contact Flight Medical for MB upgrade)

Important:

- When upgrading MB, do not remove the Jumper from the MB.

2. NEW OR DROPPED FEATURES

2.1. Dropped features

No dropped features in the release.

2.2. New features

No new features were added

3. CHANGES TO RELEASE

3.1. Automatic technical alarms threshold adjustment

A few technical alarms thresholds were adjusted to avoid false alarms (this change is transparent to the user).

3.2. Bug fix

3.2.1. UI

Capnography and Oximetry modules were wrongly detected/not detected.
This bug was fixed.

4. KNOWN ANOMALIES

See “DOC-0998 Unresolved Bugs for SW Version 1.22” (as no changes other than the specific bugs fix were made to SW 1.22).

5. OPERATING SYSTEM SUPPORTED

All of Ventoux ventilator versions.

	Ventoux SW 1.25 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1028

6. SYSTEM REQUIREMENTS

NA

7. FILES ISSUED

- flight-software-package-1.25.112.zip.gpg
- mmc-4GB
- VENTOUX Import zip file

8. DEPLOYED INSTRUCTIONS

Required equipment:

USB storage device with the new SW version (**.pkg** file)

NOTICE: two power sources are required for SW update process (AC power source and one battery or both batteries)

1. Enter the service screen with technician password.
2. Go to Tools tab.
3. Enter Software update screen.
4. Plug in the USB storage device to a USB port at the back panel of the ventilator.
5. Choose the new SW version.
6. Press Start.
7. At the end of the process, the ventilator will shut down.

9. RELATED DOCUMENTATION

- DOC-0468 A10 Ventoux Operating Manual
- DOC-0751 A06 Ventoux Service Manual
- DOC-0998 Unresolved Bugs for SW Version 1.22