

	Ventoux SW 1.27 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1064

COVER PAGE

APPROVAL				
	Name	Title	Date	Signature
Written by	Shimrit Weinberg	Product manager	Per Arena	Per Arena
Updated by	David Chen	R&D SW manager	Per Arena	Per Arena
Reviewed by	Eyal Ozeri	QA & Regulatory manager	Per Arena	Per Arena
Approved by	Amir Kleinstern	CEO	Per Arena	Per Arena

	Ventoux SW 1.27 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1064

CONTENTS

1. GENERAL 3

2. NEW OR DROPPED FEATURES 3

2.1. DROPPED FEATURES 3

2.2. NEW FEATURES..... 3

3. CHANGES TO RELEASE 3

3.1. BUG FIX 3

3.1.1 DIAGNOSTICS NOT WORKING 3

4. KNOWN ANOMALIES 3

5. OPERATING SYSTEM SUPPORTED 3

6. SYSTEM REQUIREMENTS 3

7. FILES ISSUED 3

8. DEPLOYED INSTRUCTIONS 4

9. RELATED DOCUMENTATION 4

	Ventoux SW 1.27 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1064

1. GENERAL

Important: if the current SW version of the device is 1.26 you must downgrade to SW version 1.25 first and only then upgrade to SW version 1.27.

2. NEW OR DROPPED FEATURES

2.1. Dropped features

No dropped features in the release.

2.2. New features

No new features were added

3. CHANGES TO RELEASE

3.1. Bug fix

3.1.1. Diagnostics not working

In SW version 1.26.2 the diagnostics in the service screen was not working, this bug was fixed.

4. KNOWN ANOMALIES

See “DOC-0998 Unresolved Bugs for SW Version 1.22” (as no changes other than the specific bugs fix were made to SW 1.22).

5. OPERATING SYSTEM SUPPORTED

All of Ventoux ventilator versions.

6. SYSTEM REQUIREMENTS

NA

7. FILES ISSUED

- flight-software-package-1.27.1.zip.gpg
- mmc-4GB
- VENTOUX Import zip file

 Flight Medical®	Ventoux SW 1.27 Release Note
☐ R&D ☐ QA ☐ General ☒ Production	DOCUMENT NUMBER: DOC-1064

8. DEPLOYED INSTRUCTIONS

Required equipment:

USB storage device with the new SW version (.*gpg* file)

NOTICE: two power sources are required for SW update process (AC power source and one battery or both batteries)

1. Enter the service screen with technician password.
2. Go to Tools tab.
3. Enter Software update screen.
4. Plug in the USB storage device to a USB port at the back panel of the ventilator.
5. Choose the new SW version.
6. Press Start.
7. At the end of the process, the ventilator will shut down.

9. RELATED DOCUMENTATION

- DOC-0468 A15 Ventoux Operating Manual
- DOC-0751 A08 Ventoux Service Manual
- DOC-0998 Unresolved Bugs for SW Version 1.22