

FLIGHT MEDICAL INNOVATIONS Ltd.

# FLIGHT 60T VENTILATOR Service Manual

Models: SL; DL; iO<sub>2</sub>



LIT-0113 Rev. A15

SW 5.41

# Legal Notice

## Disclaimer

FLIGHT MEDICAL INNOVATIONS Ltd. (FLIGHT MEDICAL) provides this Service Manual in its commitment to help reduce patient risk and injury. However, this manual is not intended to in any way replace or substitute duty of care to a patient, professional responsibility, or professional judgment, nor is it intended to provide any warranty, promise, guarantee, assumption of risk or duty, release, or indemnity. Physicians shall at all times maintain responsibility for patient treatment and outcomes, and FLIGHT MEDICAL further assumes no liability for patient treatment or outcome or for physician's negligence, breach of duty of care, or malpractice.

The FLIGHT 60T Ventilator operator is solely responsible for selecting the appropriate level and method of patient monitoring.

Product modification or misuse can be dangerous. FLIGHT MEDICAL disclaims all liability for the consequences of product alterations or modifications, as well as for the consequences which might result from the combination of this ventilator with other products, whether supplied by FLIGHT MEDICAL or by other manufacturers, unless such a combination has been specifically endorsed by FLIGHT MEDICAL.

The design of FLIGHT 60T Ventilator, the Operator's and Service Manuals, and the labeling on the ventilator, take into consideration that the purchase and use of the equipment is restricted to trained professionals, and that certain inherent characteristics of the ventilator are known to the operator. Instructions, warnings, and caution statements are therefore limited to the specifics of the FLIGHT 60T Ventilator.



Federal law (US) restricts this device to sale by or on the order of a physician.

This Operator's Manual excludes references to various hazards which are obvious to medical professionals and operators of this equipment, to the consequences of product misuse, and to potential adverse effects in patients with abnormal conditions.

When the FLIGHT 60T Ventilator is used in homecare and subacute environments, only properly trained personnel should operate the ventilator. The FLIGHT 60T Ventilator is a restricted medical device designed for use by respiratory therapists or other properly trained and qualified personnel under the direction of a physician and in accordance with applicable state laws and regulations.

Transport of patients with the FLIGHT 60T Ventilator requires that medical staff have a good working knowledge of the ventilator's use and problem resolution. Proper emergency backup equipment must be immediately available during transport.

FLIGHT 60T Ventilator operators must recognize their responsibility for implementing safety monitoring mechanisms which supply appropriate information on equipment performance and patient condition. Patient safety may be achieved through a wide variety of means, such as electronic surveillance of equipment performance and

patient condition. However, equipment surveillance should not replace direct observation of clinical signs.

The liability of FLIGHT MEDICAL is subject to and limited to the exclusive terms and conditions as set forth herein. Said liability is limited whether arising out of, or related to, the manufacture and sale of goods, their installation, demonstration, sales representation, use, performance, or otherwise. Any liability based upon product warranty is limited regardless of any fault attributable to FLIGHT MEDICAL and the nature of the action (including breach of warranty, negligence, and strict liability).

The written warranties are in lieu of all other warranties, expressed or implied, including, without limitation, warranties of merchantability, fitness for any purpose, or non-infringement.

FLIGHT MEDICAL shall not be liable for any special incidental or consequential damages incurred by the buyer to a third party. The buyer shall not be entitled to make liability recoveries from FLIGHT MEDICAL due to such situations.

## Warranty

The FLIGHT 60T Ventilator warranty does not apply for/ in case of:

- Defects caused by misuse, mishandling, tampering, or by modifications not authorized by FLIGHT MEDICAL or its representatives.
- Rubber and plastic components and materials, which are guaranteed to be free of defects at time of delivery.

Any product which proves during the warranty period to be defective in workmanship or material, will be replaced, credited, or repaired. FLIGHT MEDICAL retains the discretion to select the most suitable of these options. FLIGHT MEDICAL is not responsible for deterioration, wear, or abuse. In all cases, FLIGHT MEDICAL will not be liable beyond the original selling price.

Application of this warranty is subject to the following conditions:

- FLIGHT MEDICAL or its authorized representatives must be promptly notified upon detection of the defective material or equipment.
- Defective material or equipment must be returned to FLIGHT MEDICAL or its authorized representative.
- Examination by FLIGHT MEDICAL or its authorized representatives must confirm that the defect is covered by the terms of this warranty.

To ensure complete protection under this warranty, the Warranty Registration Card must be returned to a FLIGHT MEDICAL authorized representative within ten (10) days of equipment receipt.

The above is the sole warranty provided by FLIGHT MEDICAL. No other warranty, expressed or implied, is intended. Representatives of FLIGHT MEDICAL are not authorized to modify the terms of this warranty.



In no way does this or any of FLIGHT MEDICAL policies, training materials, guidelines, or instructions create an obligation for FLIGHT MEDICAL to perform any services.

# About this Document

This document is a service manual for the FLIGHT 60T and F60 Dual Limb Ventilators, ventilators that provide continuous or intermittent mechanical ventilation support for the care of individuals who require mechanical ventilation. It is intended for technicians who are responsible for maintaining, servicing, and providing troubleshooting assistance for the FLIGHT 60T and F60 Dual Limb Ventilators.

**For information on how to use the FLIGHT 60T Ventilator, see the *FLIGHT 60T Ventilator Operator's Manual*.**

## Style Conventions

Convention	Used for
Verdana	Regular text.
<b>Arial Bold</b>	Names of labels, buttons, and other elements of the user interface.
<i>Arial Italics</i>	Special terms, the first time they appear.
	<b>Notes</b> , which offer an additional explanation or a hint on how to overcome a common problem.
	<b>Warnings</b> , which indicate potentially damaging user operations and explain how to avoid them.

# Table of Contents

<b>1</b>	<b>INTRODUCTION .....</b>	<b>1-1</b>
1.1	INTENDED USE .....	1-1
1.2	SYMBOLS .....	1-1
<b>2</b>	<b>SAFETY INSTRUCTIONS .....</b>	<b>2-1</b>
2.1	GENERAL WARNINGS .....	2-1
2.2	CAUTIONS .....	2-3
<b>3</b>	<b>FUNCTIONAL DESCRIPTION.....</b>	<b>3-1</b>
3.1	FRONT PANEL .....	3-1
3.1.1	Keypad / Control Buttons .....	3-2
3.1.2	LED Indicators .....	3-3
3.2	LEFT SIDE PANEL.....	3-4
3.3	RIGHT SIDE PANEL.....	3-5
3.4	BACK PANEL .....	3-6
3.5	REMOTE ALARM CONNECTION .....	3-7
3.6	PNEUMATICS DIAGRAM .....	3-9
3.7	ELECTRICAL DIAGRAM .....	3-10
3.8	ELECTRICAL DIAGRAM – <i>IO<sub>2</sub> INTERNAL MIXER MODEL</i> .....	3-11
<b>4</b>	<b>REMOVING AND REINSTALLING MODULES .....</b>	<b>4-1</b>
4.1	INTRODUCTION .....	4-1
4.2	REQUIRED EQUIPMENT .....	4-1
4.3	REPLACING THE DETACHABLE BATTERY .....	4-2
4.3.1	Removing the Detachable Battery .....	4-2
4.3.2	Installing the Detachable Battery .....	4-2
4.4	REPLACING THE INTERNAL BATTERY .....	4-2
4.4.1	Removing the Internal Battery .....	4-2
4.4.2	Installing the Internal Battery .....	4-3
4.5	REMOVING THE VENTILATOR COVER.....	4-3
4.5.1	Removing the Ventilator Cover .....	4-3
4.5.2	Installing the Ventilator Cover .....	4-6
4.6	REPLACING THE POWER BOARD .....	4-6
4.6.1	Removing the Power Board .....	4-6
4.6.2	Installing the Power Board .....	4-8
4.7	REPLACING THE POWER SUPPLY.....	4-9
4.8	REPLACING THE FRONT PANEL ASSEMBLY.....	4-11
4.8.1	Removing the Front Panel Assembly .....	4-11

## Table of Contents

---

4.8.2	Connecting the Front Panel Assembly .....	4-13
4.9	REPLACING THE O <sub>2</sub> MIXER NEB BOARD * ( <i>IO<sub>2</sub> INTERNAL MIXER MODEL ONLY</i> ).....	4-13
4.9.1	Removing the O <sub>2</sub> mixer neb board.....	4-13
4.9.2	Installing the O <sub>2</sub> mixer board.....	4-14
4.10	REPLACING THE MAIN BOARD .....	4-14
4.10.1	Removing the Main Board .....	4-14
4.10.2	Installing the Main Board .....	4-16
4.11	REPLACING THE TOUCH SCREEN .....	4-17
4.12	REPLACING THE BLOWER ASSEMBLY.....	4-18
4.12.1	Removing the Blower Assembly .....	4-18
4.12.2	Installing the Blower Assembly .....	4-19
4.13	REPLACING THE OXYGEN SENSOR .....	4-19
4.13.1	Removing the Oxygen Sensor.....	4-19
4.13.2	Installing the Oxygen Sensor.....	4-20
4.14	REPLACING THE O <sub>2</sub> LEAK SENSOR * ( <i>IO<sub>2</sub> INTERNAL MIXER MODEL ONLY</i> ).....	4-20
4.14.1	Removing the O <sub>2</sub> leak sensor.....	4-20
4.14.2	Installing the O <sub>2</sub> leak sensor.....	4-20
4.15	REPLACING THE OXYGEN MIXER * ( <i>IO<sub>2</sub> INTERNAL MIXER MODEL ONLY</i> ).....	4-21
4.15.1	Removing the Oxygen mixer .....	4-21
4.15.2	Installing the Oxygen mixer .....	4-21
4.16	REPLACING THE O <sub>2</sub> MIXER FILTERS * ( <i>IO<sub>2</sub> INTERNAL MIXER MODEL ONLY</i> ).....	4-22
4.17	REPLACING THE PURGE BOARD ASSEMBLY .....	4-23
4.17.1	Removing the Purge Board Assembly.....	4-23
4.17.2	Installing the Purge Board Assembly .....	4-23
4.18	REPLACING THE SOLENOID BOARD ASSEMBLY .....	4-24
4.18.1	Removing the Solenoid Board Assembly.....	4-24
4.18.2	Installing the Solenoid Board Assembly.....	4-25
4.19	REPLACING THE SOLENOID ASSEMBLY .....	4-26
4.19.1	Removing the Solenoid Assembly.....	4-26
4.19.2	Installing the Solenoid Assembly.....	4-26
4.20	REPLACING THE LOWER BOARD AND D-TYPE BOARD.....	4-27
4.20.1	Removing the Lower Board and D-type Board.....	4-27
4.20.2	Installing the Lower Board and D-type Board.....	4-29
4.21	REPLACING THE FUSE.....	4-29
4.21.1	Removing the Fuse .....	4-29
4.22	REPLACING THE RUBBER BUMPERS .....	4-30
4.22.1	Removing the Rubber Bumpers.....	4-30
4.22.2	Installing the Rubber Bumpers.....	4-30
<b>5</b>	<b>SERVICE/TECHNICAL MENU .....</b>	<b>5-1</b>

5.1	SERVICE SCREEN.....	5-1
5.2	ADDITIONAL SERVICE SCREEN.....	5-2
<b>6</b>	<b>SOFTWARE UPGRADE.....</b>	<b>6-6</b>
6.1	INSTALLING THE SW UPGRADE.....	6-6
6.2	CALIBRATION INTEGRITY CHECK.....	6-6
<b>7</b>	<b>CALIBRATION AND VERIFICATION TESTS .....</b>	<b>7-8</b>
	REQUIRED EQUIPMENT .....	7-8
7.1	CALIBRATION .....	7-8
7.1.1	Pressure Sensors Calibration .....	7-8
7.1.2	Pressure Relief Calibration.....	7-11
7.1.3	Bias Flow Calibration .....	7-12
7.1.4	Oxygen Sensor Calibration .....	7-13
7.1.5	O <sub>2</sub> Leak Sensor calibration* ( <i>iO<sub>2</sub> Internal mixer model only</i> )....	7-14
7.2	OPERATIONAL VERIFICATION PROCEDURE (OVP) .....	7-15
7.2.1	Standard Ventilator Settings (STS).....	7-15
7.2.2	Front Panel Self Test .....	7-15
7.2.3	Circuit Test .....	7-16
7.2.4	Volume verification .....	7-17
7.2.5	Pressure Verification.....	7-17
7.2.6	Bias Flow .....	7-18
7.2.7	PEEP .....	7-18
7.2.8	Pressure Trigger .....	7-19
7.2.9	Exhalation Valve Leak.....	7-20
7.2.10	High Pressure Alarm.....	7-21
7.2.11	Battery /Charger /Power Supply .....	7-22
7.2.12	Buzzer Test .....	7-23
7.2.13	Oxygen Calibration Test.....	7-23
7.2.14	Nebulizer compensation test * ( <i>iO<sub>2</sub> Internal mixer model only</i> )	7-24
7.2.15	OVP Test Results.....	7-25
7.3	TEST TABLE .....	7-27
<b>8</b>	<b>MAINTENANCE.....</b>	<b>8-1</b>
8.1	SCHEDULED MAINTENANCE.....	8-1
8.2	DOWNLOAD LOG FILES .....	8-1
<b>9</b>	<b>TROUBLESHOOTING GUIDE .....</b>	<b>9-3</b>
9.1	INTRODUCTION .....	9-3
9.2	ERROR MESSAGES .....	9-3
<b>10</b>	<b>REPACKAGING AND SHIPPING .....</b>	<b>10-1</b>
	RGA RETURNS .....	10-1
<b>11</b>	<b>SPARE PARTS.....</b>	<b>11-2</b>
11.1	ELECTRONIC .....	11-2

**Table of Contents**

---

11.2 MECHANICAL / ASSEMBLIES ..... 11-5  
11.3 CABLES ..... 11-7










# 1 Introduction

This Service Manual (V60-00002-18 Rev. A) provides information for servicing the FLIGHT 60T Ventilator. It is for use by authorized service personnel while installing, servicing, and repairing the ventilator.













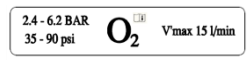
## 1.1 Intended Use

The FLIGHT 60T Ventilator is intended to provide continuous or intermittent mechanical ventilation support for the care of individuals who require mechanical ventilation. Specifically, the FLIGHT 60T Ventilator is applicable for adult and pediatric patients, greater than or equal to 10 kg (22 lbs).

## 1.2 Symbols

Symbol	Description
<b>Front Panel</b>	
	<b>On/Off</b>
	<b>Alarm Reset</b>
	<b>OK (Enter)</b>
	<b>Decrease Button</b>
	<b>Increase Button</b>
	<b>Cancel</b>
	<b>Panel Lock</b>
	<b>Manual Breath</b>
	<b>Parameters Screen</b>
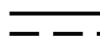
## Symbols

Symbol	Description
	Extended Screen
	Technical Screen
	Nebulizer Port
<b>Rear Panel</b>	
	Caution; consult accompanying documents
	Type BF applied part
	Temperature limitation
	Humidity limitation
	Atmospheric pressure limitation
	DC – Direct Current
	AC – Alternating Current
	USB – Universal Serial Bus
	LAN – Local Area Network
	High Pressure and Low-Flow Oxygen Port

## 2 Safety Instructions

At all times, strictly follow this manual. The safe use of the FLIGHT 60T Ventilator requires full understanding of its operation, and adherence to the manual's instructions. The equipment is only to be used for the purpose specified in section 1.1. Observe all of the WARNINGS and CAUTIONS posted in this manual, and on buttons found on the FLIGHT 60T Ventilator and associated accessories.

### 2.1 General Warnings



External power connection: To maintain grounding integrity when using AC power, only connect to grounded receptacles. Always disconnect the external power prior to servicing. There is a risk of explosion if used in the presence of flammable anesthetics.



All settings and adjustments in the different ventilation modes must be made in accordance with a physician's prescribed therapy.



Do not use electrically conductive patient circuits.



Always use a clean, disinfected patient circuit.



Always use an outlet filter or equivalent at the Airway Pressure Connector, to protect the internal sensors from moisture and other contaminants.



The ventilator is ready for operation only when:

1. It is completely assembled.
2. The OVP has been successfully completed.



Failure to identify and correct alarm violations may result in patient injury.

General Warnings



Ensure that the oxygen source is not empty before and during the use of the optional Air/Oxygen Entrainment Mixer or Oxygen Blending Bag Kit.



As Li-Ion batteries are charged and discharged over time, their ability to hold a charge is decreased with use. This can shorten the amount of time the ventilator can function while on battery power.



The batteries should be replaced when the batteries no longer meet the needs of the user. This depends on a number of factors including settings and usage patterns.



Charge the batteries for a minimum of three hours before powering the ventilator from the batteries. This provides fully charged batteries.



During storage, charge the batteries for a minimum of three hours every 30 days. This provides charged batteries.



Always ensure that the green Ext. Power LED is illuminated after connecting the FLIGHT 60T Ventilator to an external AC or DC power source. If the LED is not illuminated, check all power connections and resolve any problems.



Always plug the FLIGHT 60T Ventilator into an AC power supply source when not in use, to ensure best battery performance.



The flow resistance of the air inlet filter, located on the right side of the ventilator, is likely to increase with repeated use. Ensure that the filter is changed regularly.



Only a FLIGHT MEDICAL approved patient circuit can be used with the FLIGHT 60T Ventilator.



When using a D.L ventilator with a single patient circuit, Bias Flow calibration with a single patient circuit must be performed.



Only a FLIGHT MEDICAL approved exhalation valve can be used with the FLIGHT 60T Ventilator.



Perform a Circuit Test each time a clean circuit/exhalation valve is installed.



This FLIGHT 60T Ventilator has been tested and found to comply with the EMC limits according to the EN60601-1-1-2 standard class B. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates and can radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Reorient or relocate the receiving device.

Increase the distance between the equipment.

Connect the equipment into an outlet on a circuit different from that to which the device (s) is connected.

Consult the manufacturer for help.

## 2.2 Cautions



Only use medical grade oxygen with the Air/Oxygen Entrainment Mixer or Oxygen Blending Bag Kit.



Do not place liquid containers in the immediate vicinity or on top of the FLIGHT 60T Ventilator. Liquids that get into the ventilator can cause equipment malfunction and damage.

Cautions



After the FLIGHT 60T Ventilator is serviced, it must completely pass an Operational Verification Procedure (OVP) before being returned to patient use.



An authorized FLIGHT MEDICAL factory-trained technician must do all service or repairs performed on the FLIGHT 60T Ventilator.



Do not open the ventilator or perform service on an open unit while connected to external power.



Use standard antistatic techniques while working inside the ventilator or handling any electronic parts.



Clean all external accessories of the ventilator prior to servicing.



Water in the oxygen supply can cause equipment malfunction and damage.



Batteries contain Li-Ion. Do not discard them in an incinerator or force them open. Batteries should not be disposed of with normal waste.



Use the tools and equipment specified in this manual to perform specific procedures.

### 3 Functional Description

#### 3.1 Front Panel

The front panel contains the control buttons, visual indicators, display screen, and patient circuit connection. There are 2 types of keypad – symbols or print.

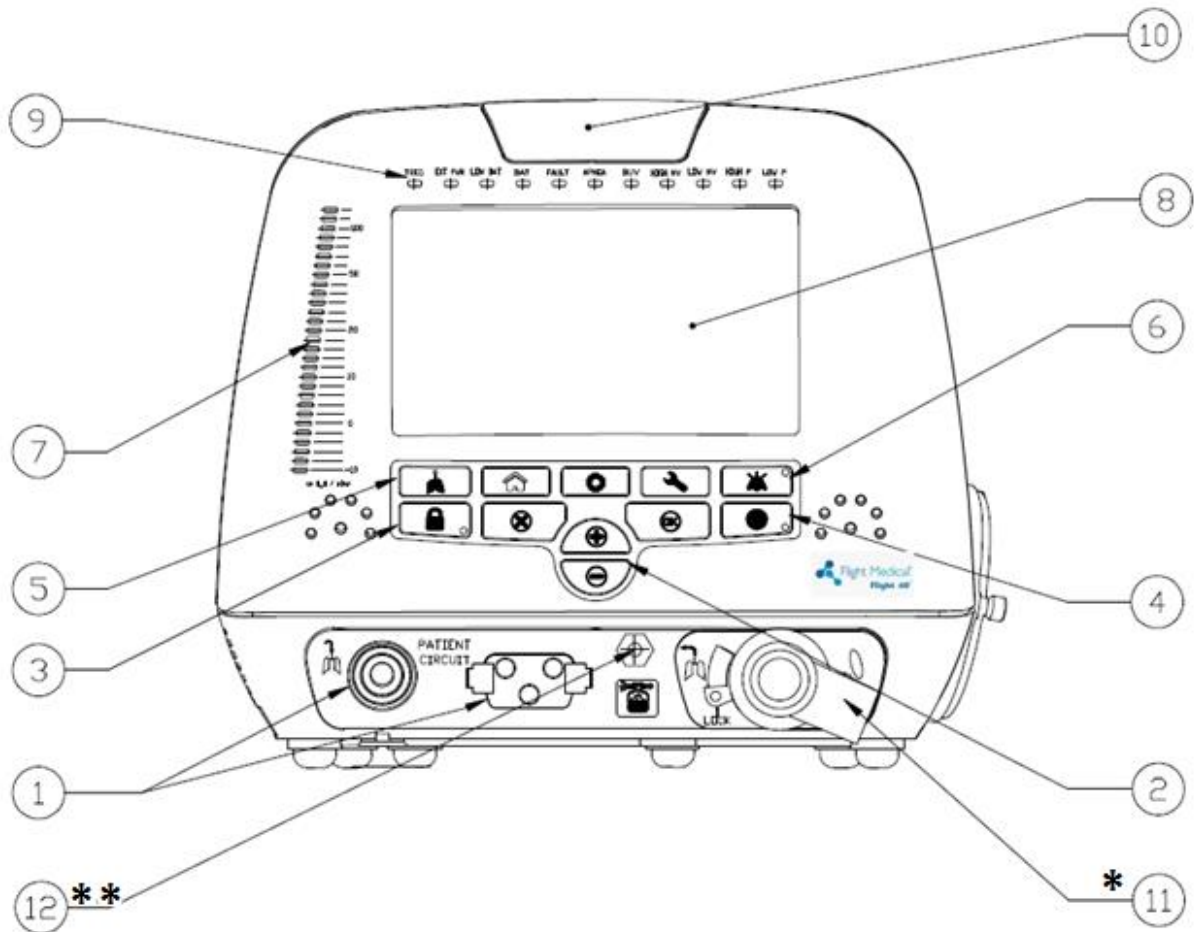


Figure 1 – Front Panel

Label	Name	Description
1	Patient Circuit Connector	Composed of a gas outlet and quick connector.
2	+/- button	Enables the user to adjust setting parameters.
3	Panel Lock button	Enables the user to lock the ventilator’s control, preventing accidental changes. Pressing the button of a locked panel and then Enter, unlocks the panel.
4	On/Off button	Turns the ventilator on or off, to start or stop ventilation.
5	Manual Breath button	Delivers a user initiated manual inflation.

## Functional Description

### Front Panel

Label	Name	Description
6	Audio Paused / Alarm Reset button	Toggle button. Pressing Audio Paused temporarily silences the audible alarm; pressing Alarm Reset clears lit alarm LEDs.
7	Pressure Gauge	The pressure gauge is a visual indicator of breath activity, which shows the dynamic movements of the breath pressures. When a breath is being delivered, the user can see the relative pressure and phase of the breath (inspiration or expiration).  The pressure gauge is comprised of 29 LEDs. From -10 to +20 cmH <sub>2</sub> O, each notch equals 2 cmH <sub>2</sub> O; from 20 to 50 cmH <sub>2</sub> O, each notch equals 5 cmH <sub>2</sub> O; above 50 cmH <sub>2</sub> O, each notch equals 10 cmH <sub>2</sub> O.
8	Display touch screen	Enables the user to modify the ventilation, alarm, and technical settings, and to view real time patient data, alarms, battery status and logs.
9	LED Indicators	Inform the user of various events.
10	Primary Alarm LED	Flashes red to indicate that there is a high priority alarm.
11*	Dual Limb Exhalation Valve (DL model only)	Connects the patient circuit expiratory limb.
12**	Nebulizer Port (iO <sub>2</sub> Internal mixer model only)	Connects to the pneumatic nebulizer.

### 3.1.1 Keypad / Control Buttons

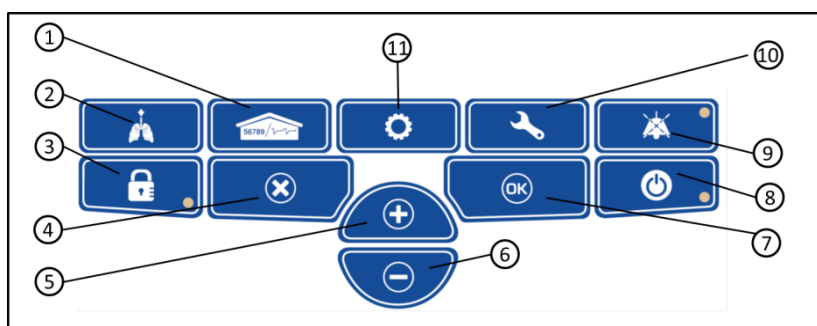












Figure 2 – Keypad / Control buttons

Item	Symbol	Description
<b>1 – Parameters (home)</b>		The Parameters screen is the Flight 60T's default screen. Display switches automatically to Parameters from the other screens if not operated for 30 seconds.  Use the Parameters button to toggle between the numeric and the graphic displays.

Item	Symbol	Description
<b>2 - Manual Breath</b>		Delivers a user initiated manual inflation.
<b>3 - Panel Lock</b>		Enables the user to lock the ventilator's control, preventing accidental changes. Pressing the button of a locked panel and then Enter, unlocks the panel.
<b>4 - Cancel</b>		Enable the user to cancel parameters change.
<b>5 - Increase Button</b>		Enables the user to adjust setting parameters upwards.
<b>6 - Decrease Button</b>		Enables the user to adjust setting parameters downwards.
<b>7 - OK (Enter)</b>		Enable the user to confirm parameters or mode change.
<b>8 - On/Off</b>		Turns the ventilator on or off, to start or stop ventilation.
<b>9 - Alarm Reset</b>		The Alarm Reset silences the audible alarm and clears lit alarm LEDs.
<b>10 - Technical</b>		Technical data and selection options.
<b>11 - Extended</b>		Additional ventilation parameters screen.

### 3.1.2 LED Indicators

The LED indicators on the front panel inform the user of various events.

The following table describes the available LED indicators.

LED Indicator	Description
<b>TRIG</b>	Green LED indicates a patient's breathing effort.
<b>EXT PWR</b>	Yellow LED indicates that an external power source is being applied to the ventilator.
<b>LOW BAT</b>	Red LED indicates that total batteries charge level is below 50%.
<b>BAT</b>	Orange LED indicates that the ventilator is powered on batteries.
<b>FAULT</b>	Red LED indicates a ventilator malfunction.
<b>APNEA</b>	Red LED indicates that no breaths have been delivered for the preset APNEA interval.
<b>BUV</b>	Red LED indicates that backup ventilation is active.
<b>HIGH MV</b>	Red LED indicates that the high inspiratory minute volume alarm limit is being violated.
<b>LOW MV</b>	Red LED indicates that the low inspiratory minute volume alarm limit is being violated.
<b>HIGH P</b>	Red LED indicates that the high peak airway pressure alarm limit is being violated.

## Functional Description

---

### Left Side Panel

LED Indicator	Description
LOW P	Red LED indicates low peak airway pressure.

---

## 3.2 Left Side Panel

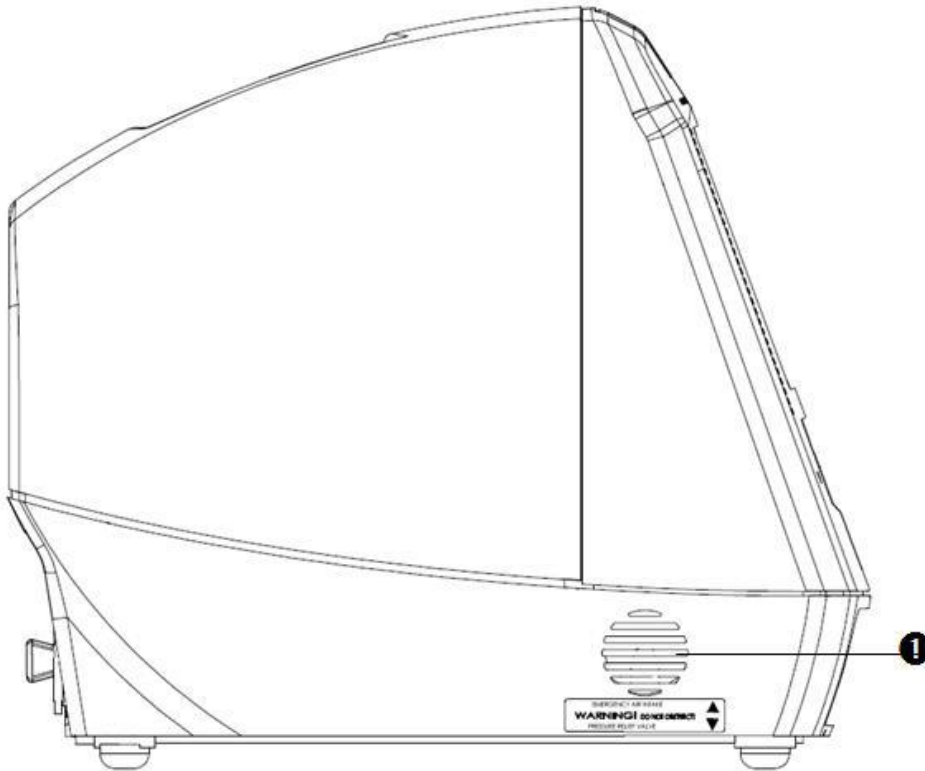


Figure 3 – Left Side Panel

Label	Name	Description
1	Emergency Air Intake	Enables the patient to pull ambient air into the patient circuit in the event of a complete system failure. The Air Intake opening pressure is approximately -3 cmH <sub>2</sub> O (-3 mbar).

---

### 3.3 Right Side Panel

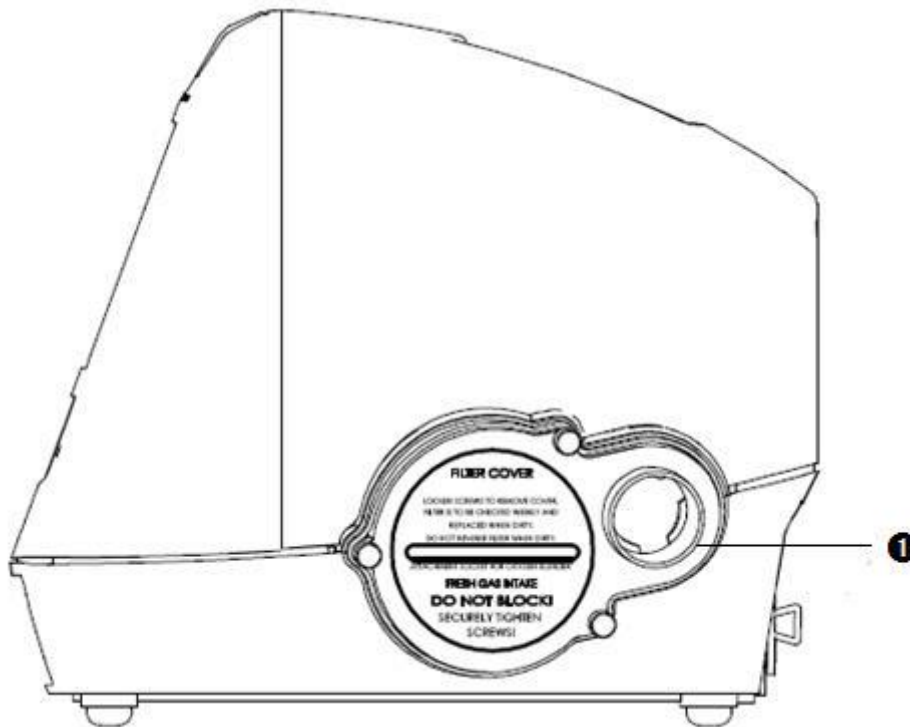


Figure 4 - Right Side Panel

Label	Name	Description
1	Fresh Gas Intake and Filter Cover	Environmental air enters through this 30 mm ID Fresh Gas Intake. The air inlet particle filter is placed behind the Filter Cover to protect the patient as well as the ventilator's piston system from dirt and particles. The Fresh Gas Intake also serves as the attachment socket for the optional FLIGHT 60T Ventilator Air/Oxygen Entrainment Mixer or Oxygen Blending Bag.

Back Panel

### 3.4 Back Panel

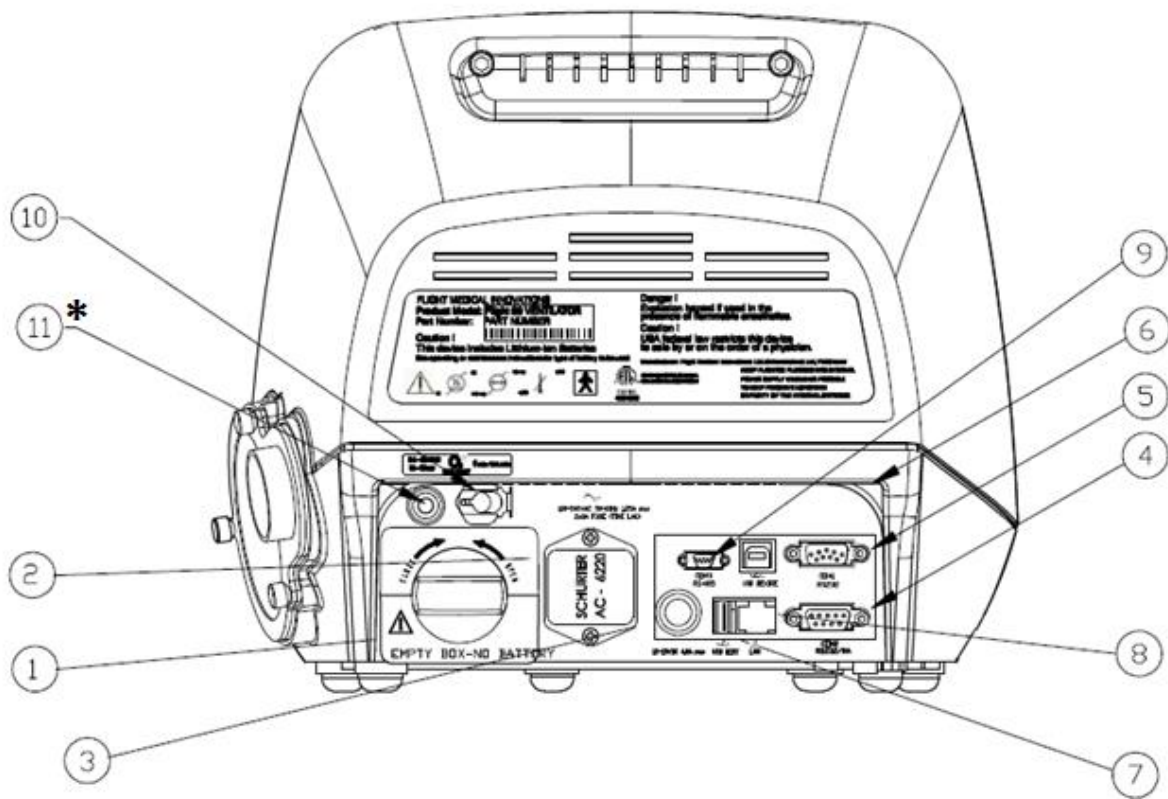


Figure 5 – Back Panel



To ensure proper grounding and prevent possible shock hazards, this device should only be connected to grounded power receptacles.

Label	Name	Description
1	Detachable Battery	Li-Ion 14.8 VDC
2	AC Connector with Fuses	100 – 240 V AC, 50 – 60 Hz, Fuses 2x8A (time lag)
3	DC Connector	12.5 – 15 V DC
4	RS-232 Serial Port (COM2)	Remote alarm connector (Normally Open and Normally Closed options).
5	RS-232 Serial Port (COM1)	Online output of events and error messages to the PC, using a dedicated PCS2 protocol; for authorized and qualified service technicians only.
6	USB B type	PC connector: USB port for downloading the main application from the PC using a dedicated PCS2 protocol; for authorized and qualified service technicians only.

<b>Label</b>	<b>Name</b>	<b>Description</b>
<b>7</b>	USB A type	USB port for uploading LOG files to an external memory stick; for authorized and qualified service technicians only.
<b>8</b>	LAN (RJ45)	LAN for network logging (currently not available).
<b>9</b>	Mini RS-485 (COM3)	For connecting FLIGHT MEDICAL peripherals. For future use.
<b>10</b>	Low Flow Oxygen Port	Low flow oxygen enrichment source.
<b>11*</b>	High Pressure O <sub>2</sub> Port ( <i>iO<sub>2</sub></i> <i>Internal mixer model only</i> )	Connects to high pressure O <sub>2</sub> .

### 3.5 Remote Alarm Connection

The Flight-60 ventilator can be connected to third party remote alarm system. The remote alarm station displays all visible and audible alarms and alerts if the ventilator is shutdown.

Remote alarm cable should be connected to the COM2 RS232/RA port (the lower port) in the back of the ventilator. The COM2 port provides a dry contact output, both Normally Open (NO) and Normally Closed (NC).


- Normally Open – Pin 7 – closing the contacts to transmit an alarm.
- Normally Closed – Pin 8 – opening the contacts to transmit an alarm.

The dry contacts current should not exceed 1mA.


The voltage on the contacts must be lower than 12 Volts.




### 3.6 Pneumatics Diagram

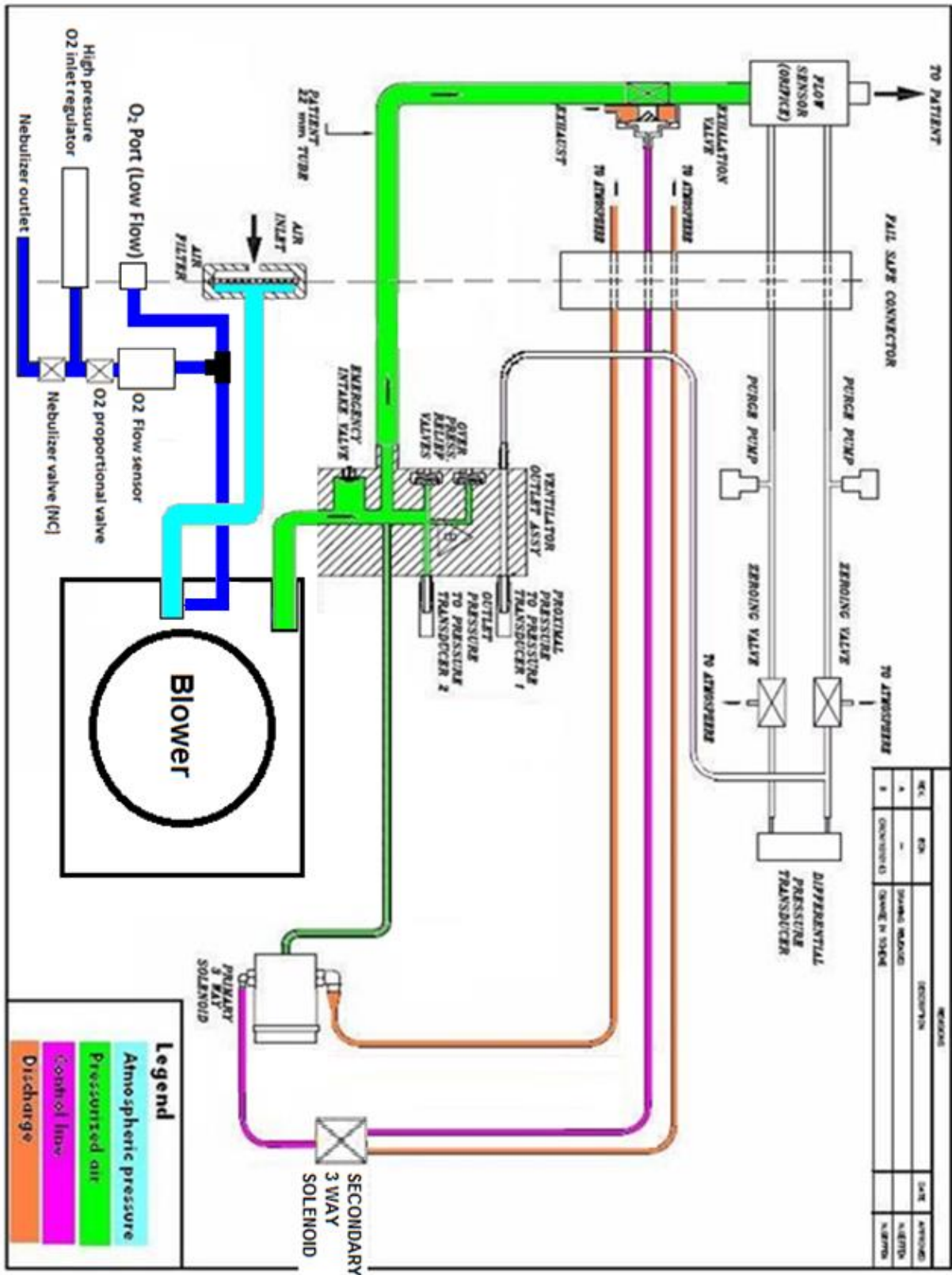

### 3.7 Electrical Diagram

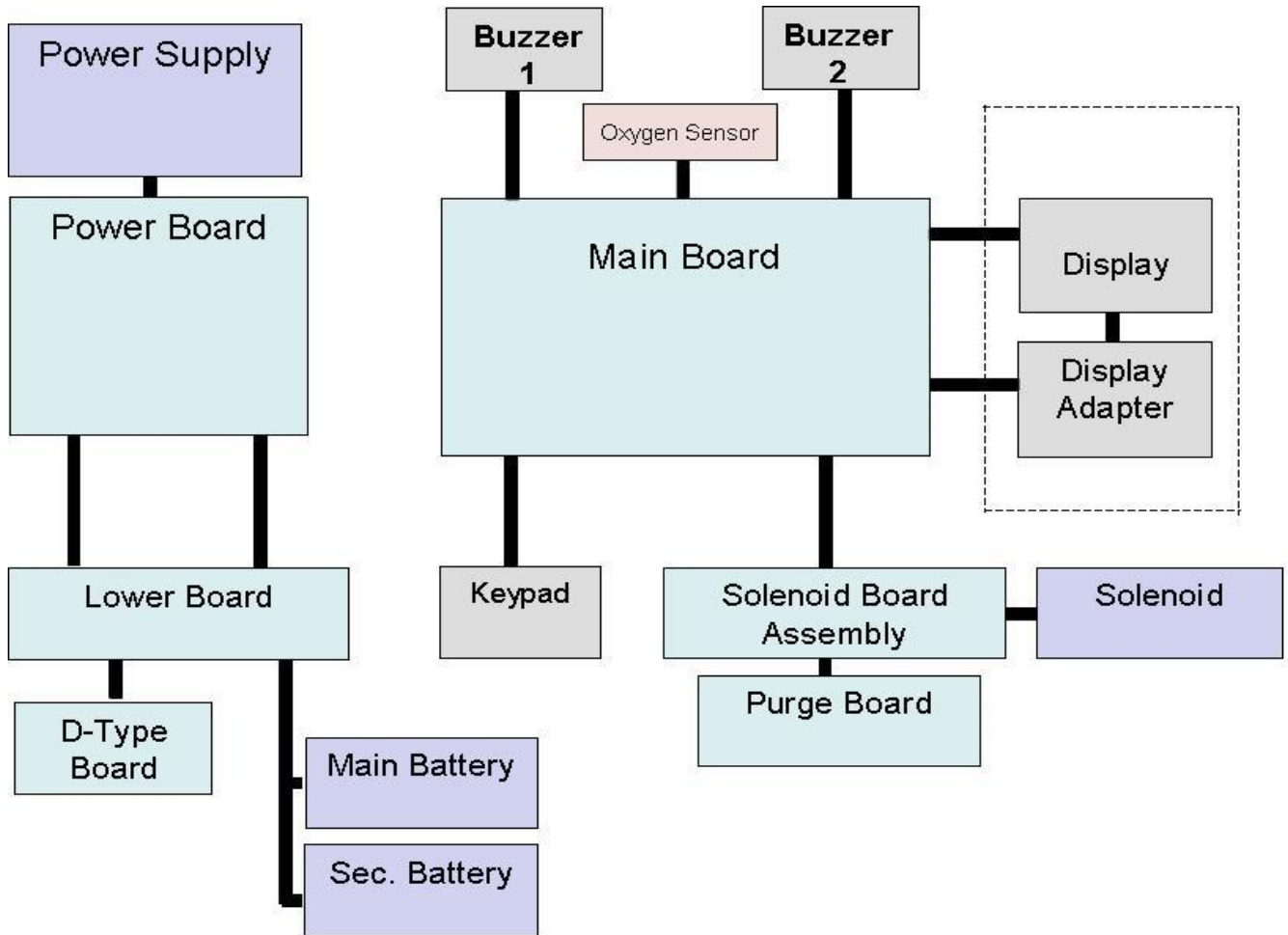

### 3.8 Electrical Diagram – iO<sub>2</sub> Internal Mixer model

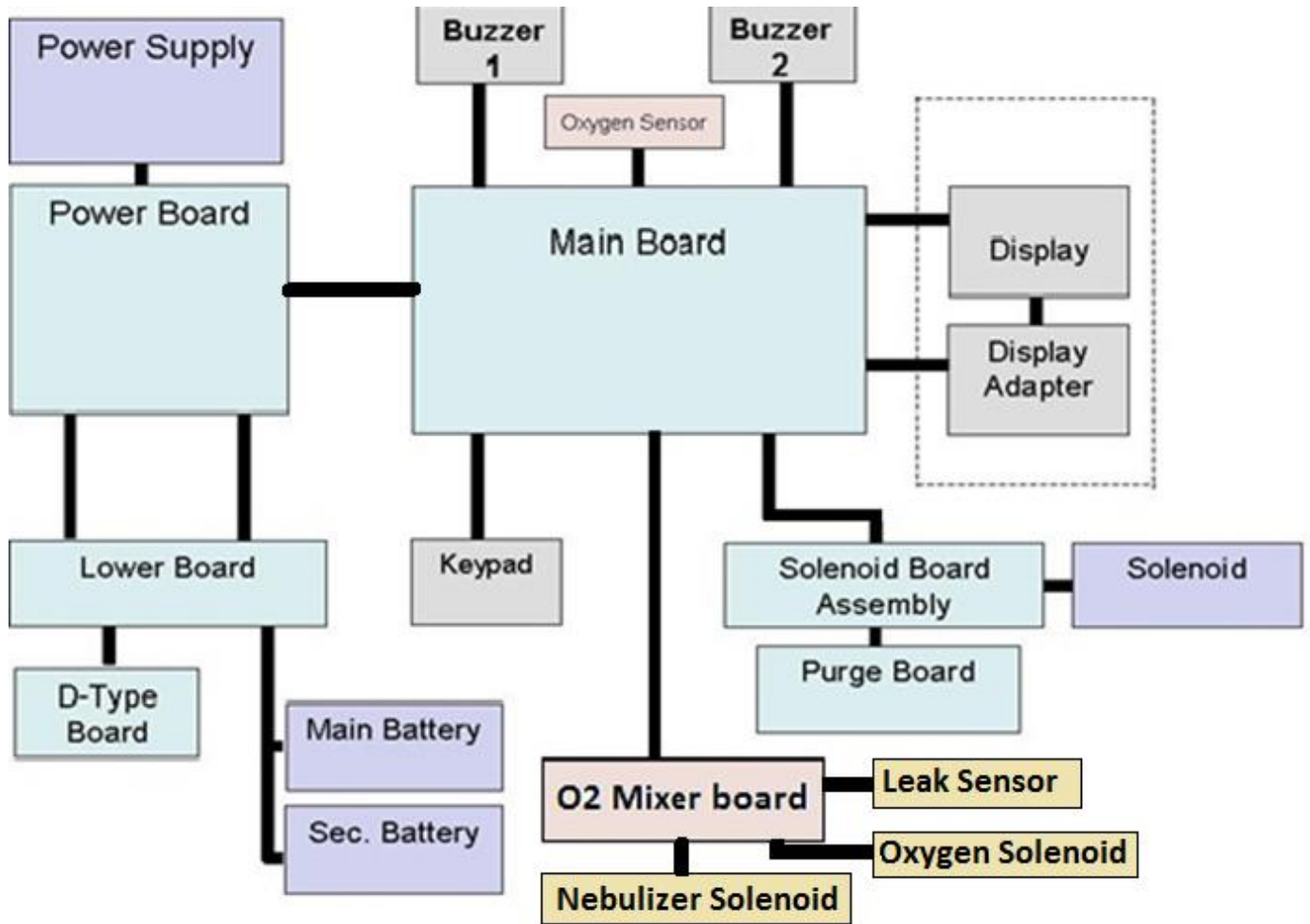



## 4 Removing and Reinstalling Modules

### 4.1 Introduction

This chapter provides detailed procedures on how to remove FLIGHT 60T Ventilator modules and assemblies that you suspect are worn, damaged, or performing improperly, and how to install new or repaired modules and assemblies.

**Make sure you are familiar with the FLIGHT 60T Ventilator's operation before attempting any service or maintenance.**



**Ventilator log files must be downloaded prior to any service attempt.** Refer to the Maintenance chapter for instructions.



**Hazardous voltages are present inside the ventilator. Disconnect electrical power and oxygen sources before attempting any disassembly. Remove watch and any jewelry. Failure to do so may result in injury to service personnel or damage to the ventilator.**

### 4.2 Required Equipment

The following equipment is required to remove and replace FLIGHT 60T ventilator modules:

- Antistatic wrist strap
- Phillips screwdriver PH1
- Flat screwdriver 0.8x4
- Torx screwdriver BT10x200
- Cutter
- Cable ties: PLT1M-M69; PLT4S-M69



To protect external parts and the touch screen from any damage while in contact with the working surface, place the ventilator on a cushioned surface while it is being serviced.



After servicing the ventilator modules, refer to the Test Table in the Operational Verification Procedure (OVP) chapter and perform the necessary Operation Verification Tests.

## 4.3 Replacing the Detachable Battery

### 4.3.1 Removing the Detachable Battery

➔ To remove the detachable battery:

1. Turn the lock dial counterclockwise, in the direction of the OPEN arrow.



Figure 6 – Removing the Detachable (Main) Battery

2. Pull the battery out of the ventilator.

### 4.3.2 Installing the Detachable Battery

➔ To install the detachable battery:

1. Insert the detachable battery into the ventilator.
2. Turn the lock dial clockwise, in the direction of the CLOSE arrow, until it is firmly locked.

## 4.4 Replacing the Internal Battery

### 4.4.1 Removing the Internal Battery

➔ To remove the internal battery:

1. Remove the four Phillips screws that secure the battery panel to the bottom panel.



*Figure 7 – Removing the Internal Battery Panel*

2. Remove the internal batteries from the battery compartment.

#### **4.4.2 Installing the Internal Battery**

- ➔ **To install the internal battery:**
  - Follow the above procedure in reverse order.

### **4.5 Removing the Ventilator Cover**

#### **4.5.1 Removing the Ventilator Cover**

The following procedure describes how to remove the ventilator cover. In most cases the cover must be removed in order to access the ventilator modules.

- ➔ **To remove the ventilator cover:**
  1. Remove the two Torx screws inside the ventilator handle.

Removing the Ventilator Cover



*Figure 8 – Removing Screws Inside Handle*

2. Remove the two Torx screws connecting the ventilator cover to the back panel.



*Figure 9 – Removing Back Screws*

3. Remove the two Torx screws from the bottom panel.

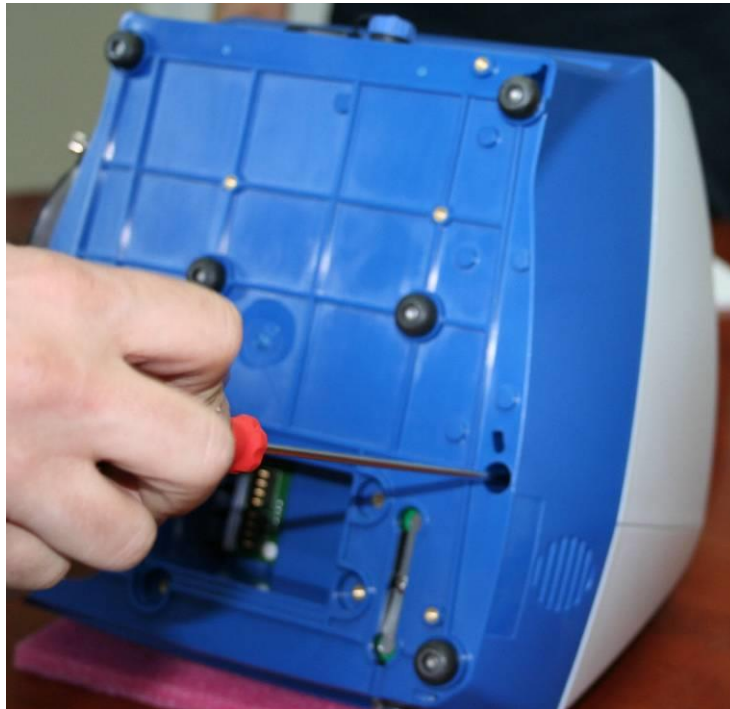


Figure 10 – Removing the Bottom Panel Screws

4. Remove the ventilator cover.

The modules of the ventilator are now exposed and accessible.



Replacing the Power Board

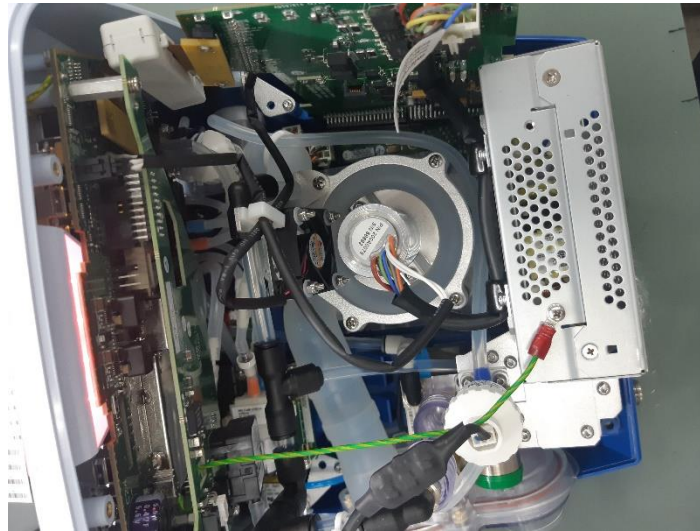


Figure 11 – Removing the Ventilator Cover



Cover screws are longer than the rest of the screws used in the ventilator

## 4.5.2 Installing the Ventilator Cover

- ➔ To reattach the ventilator cover:
  - Follow the above procedure in reverse order.

## 4.6 Replacing the Power Board

### 4.6.1 Removing the Power Board

- ➔ To remove the power board:
  1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
  2. Remove the ventilator cover (see section 4.5).
  3. Disconnect the flat flex cable (1), connectors 2, 3, 4 from the power board


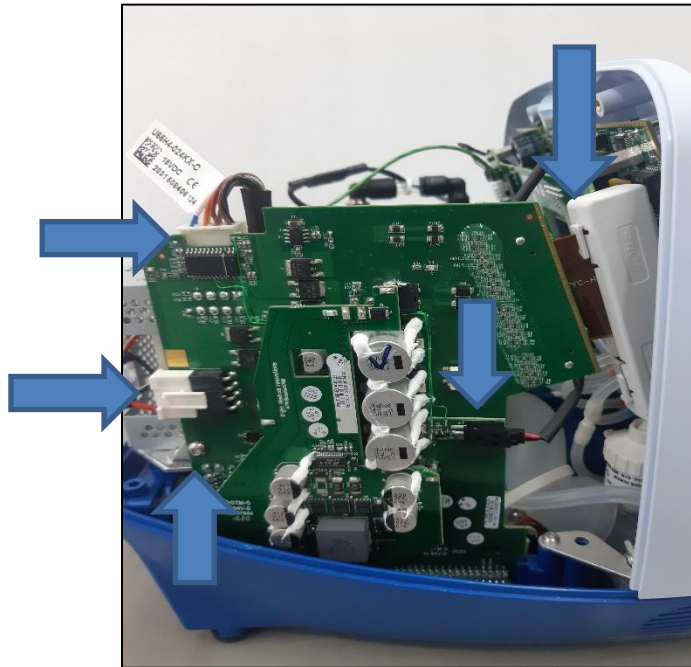


Figure 12 – Disconnecting the Power Board

4. Remove the Torx screw (5) connecting the power board to its bracket.



**Do not hold the power board by its components.** To pull the board out, hold the grooves. Refer to the figure below for correct positioning.

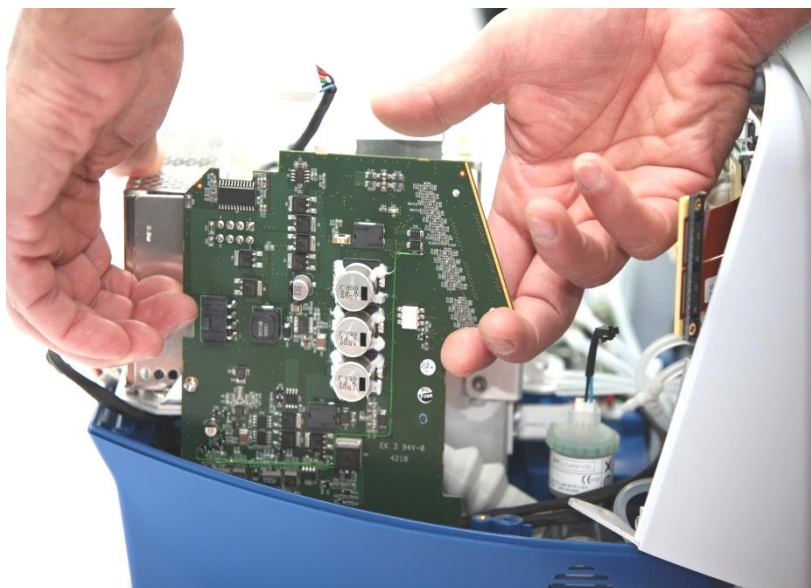

## Replacing the Power Board


*Figure 13 – Lifting Power Board from Ventilator Base*

4. Holding the board by the grooves, carefully pull the board out by lifting up slightly on one side and then the other, continuing to alternate sides until the board is released from the socket.


### 4.6.2 Installing the Power Board

➔ **To install the power board:**

1. Check that the power board sockets do not have any bent pins.
2. Slide the power board through the right and left guide tracks, positioning the power board connectors over the sockets.

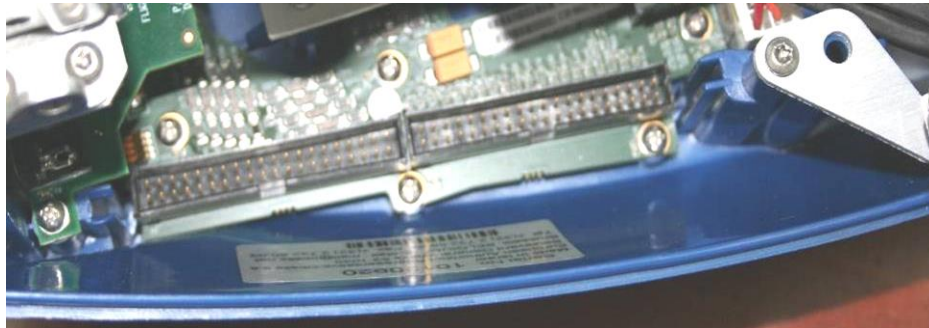

*Figure 14 – Power Board Slots and Tracks*

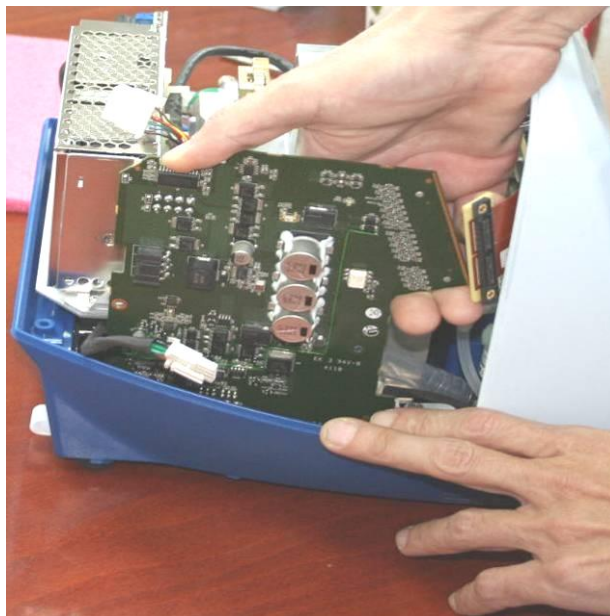

*Figure 15 – Inserting Power Board into Ventilator Base*

3. Firmly push down the power board, making sure that the power board connectors fit well into the sockets (pin headers) in the base of the ventilator.

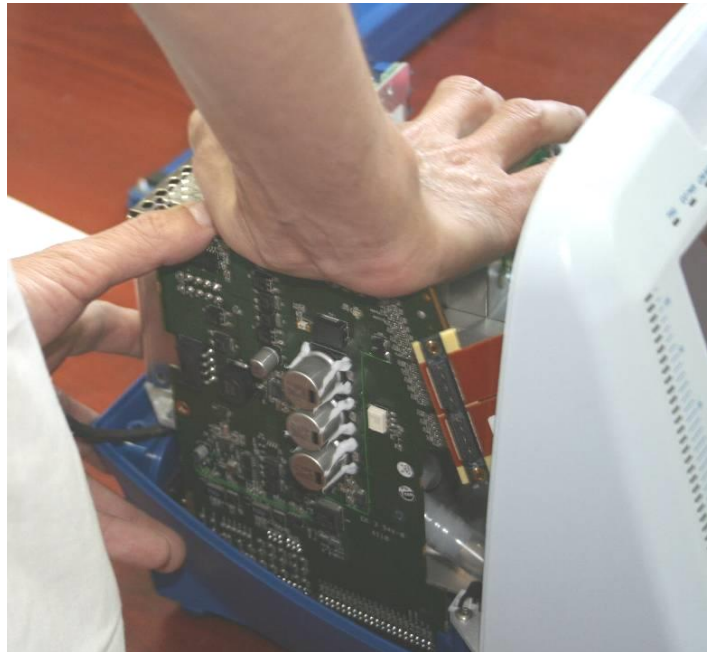


Figure 16 – Pushing Power Board into Ventilator Base

4. Perform steps 1 to 6 of the removal procedure in the reverse order.

## 4.7 Replacing the Power Supply

### ➔ To remove the power supply:

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Remove the screw from the grounding cable.
4. Release the Blower motor cable from the cable clamps on the power supply.
5. Disconnect the 2 cables from the Power Supply:

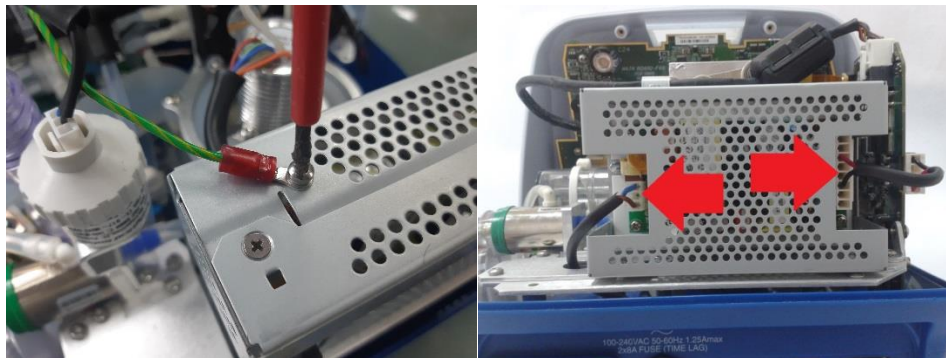


Figure 17 – Power Supply cables

## Removing and Reinstalling Modules

### Replacing the Power Supply




Disconnect the output cable from the Power Board first. Use flat/needle nose pliers to gently pull out the connector from the Power Supply

6. Remove the 4 Torx screws securing the bracket



*Figure 18 – Securing screws*

7. Remove the 4 Philips screws securing the bracket to the Power Supply

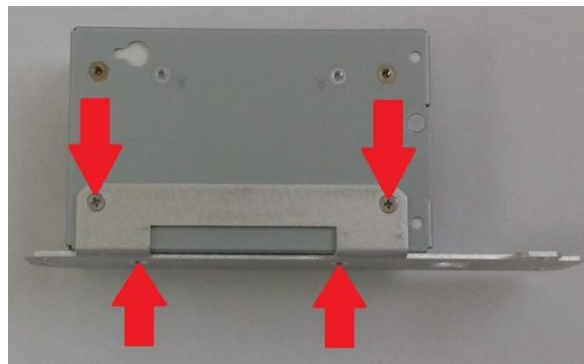

*Figure 19 – Detaching the Power Supply from Its Bracket*

➔ **To install the power supply:**

Follow the above procedure in reverse order

## 4.8 Replacing the Front Panel Assembly

### 4.8.1 Removing the Front Panel Assembly

➔ To remove the front panel assembly:

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Remove screw of the grounding cable from power supply.
4. Disconnect the flat flex cable P16 from the Main Board.

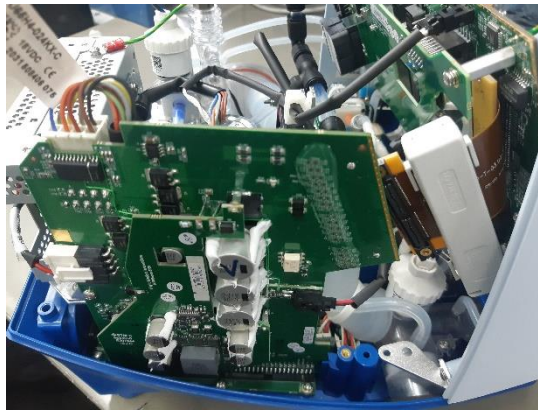


Figure 20 – Flat Flex Cable going to P16

5. On the underside of the ventilator, unscrew the three Torx screws securing the bottom of the front panel.



Figure 21 – Loosening the Torx Screws Under the Front Panel

Replacing the Front Panel Assembly

6. Remove the Torx screws from the front panel brackets (one on each side). Carefully slide the front panel outwards to allow access to electrical cables and silicon tubes.



Figure 22 – Torx Screws on Front Panel Bracket

**For iO<sub>2</sub> Internal Mixer model**

6. Disconnect the 2 silicon tubes from the O<sub>2</sub> mixer neb board.
7. Disconnect the nebulizer cable from the O<sub>2</sub> mixer neb board.
8. Disconnect the mixer cable from the O<sub>2</sub> mixer neb board.
9. Disconnect the O<sub>2</sub> leak sensor cable from the O<sub>2</sub> mixer neb board.

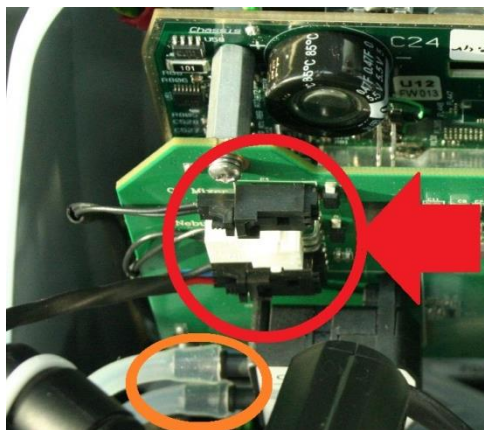


Figure 23 – Disconnecting from O<sub>2</sub> mixer neb board

10. Disconnect the 3 silicon tubes coming from the main board. Note the color coding. Disconnect them from the Y connectors and not from the board itself.
11. Disconnect the Blower cable.
12. Disconnect the solenoid board flat cable.
13. Disconnect the O<sub>2</sub> sensor cable.
14. Remove the front panel from the ventilator.

## 4.8.2 Connecting the Front Panel Assembly

- ➔ To connect the front panel assembly:
  - Follow the above procedure in reverse order.


## 4.9 Replacing the O<sub>2</sub> mixer Neb board \* (iO<sub>2</sub> Internal mixer model only)

### 4.9.1 Removing the O<sub>2</sub> mixer neb board

- ➔ To remove the O<sub>2</sub> mixer neb board:
  1. Follow the procedures for removing the front panel assembly (see section 4.8 )
  2. remove the 3 screws securing the O<sub>2</sub> mixer Neb board to the Main Board.

Figure 24 – Securing screws

3. Pull the mixer board out of the connector .

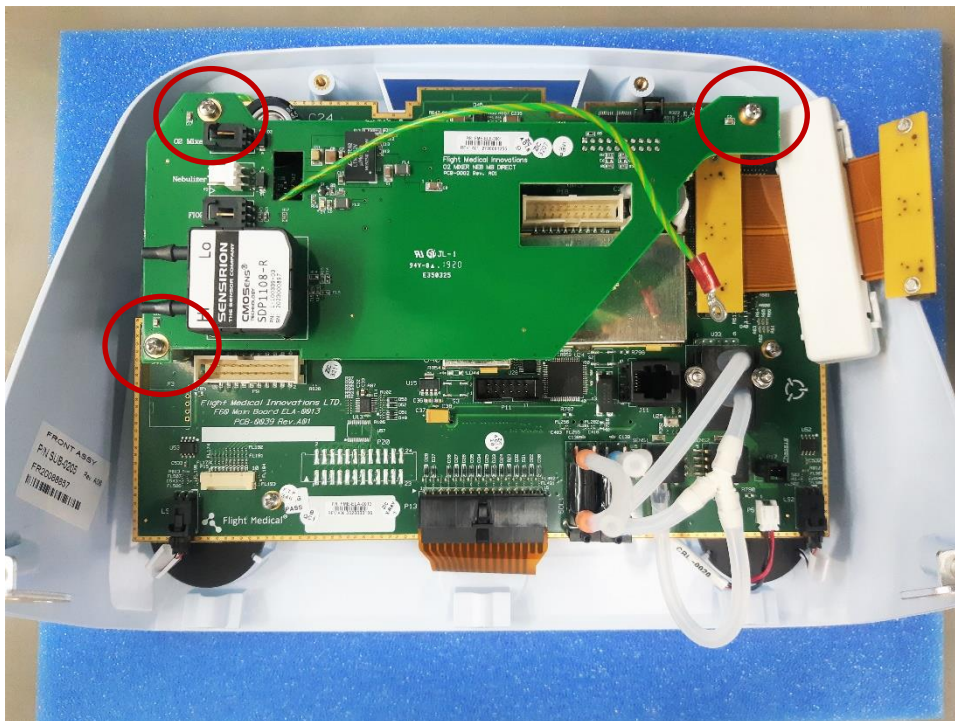

Replacing the Main Board

### 4.9.2 Installing the O<sub>2</sub> mixer board

- ➔ To connect the O<sub>2</sub> mixer board:
  - Follow the above procedure in reverse order.

## 4.10 Replacing the Main Board



Record system S/N, Blower S/N and motor hours prior to replacing the main board. After replacing the board need to update the SW with those numbers, as described in chapter 8.

### 4.10.1 Removing the Main Board

- ➔ To remove the main board:

1. **For iO<sub>2</sub> internal mixer model** - Follow the procedure for Removing the O<sub>2</sub> mixer neb board (see section 4.9)
2. Remove the 6 Torx screws holding the Main Board to the front panel
3. On the bottom side of the Main Board, disconnect buzzer connectors LS1 and LS2.
4. Disconnect the back light cable from connector P5.



Do not pull the wires. Press the locker and then remove.

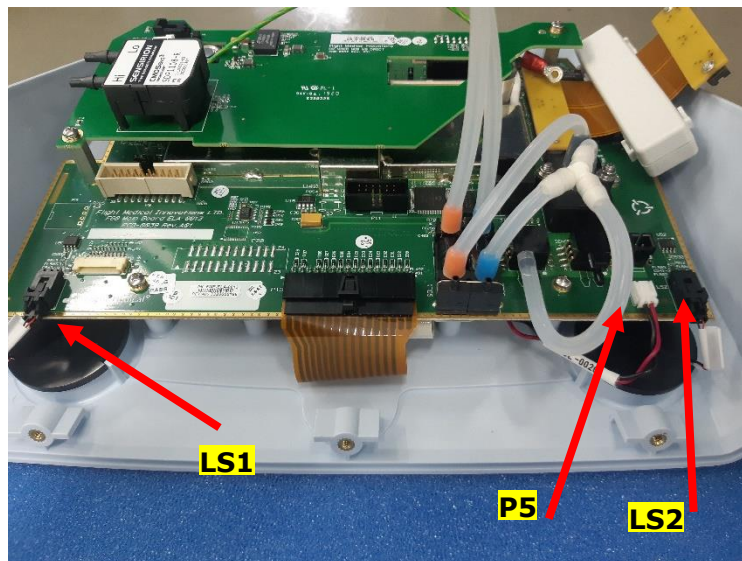


Figure 25 – LS1; LS2; P5 connectors



Figure 26 – Disconnecting LS2 connector

5. Disconnect the P13 flat cable by pressing hard on the locker and then pulling it out.



Figure 27 – Disconnecting the P13 Flat Cable


6. Remove the six Philips screws securing the Main Board to the front panel.



Replacing the Main Board

*Figure 28 – Detaching the Main Board from the Front Panel*

7. Remove the grounding screw at the top left of the board.
8. Remove the grounding screw at the top right of the board.

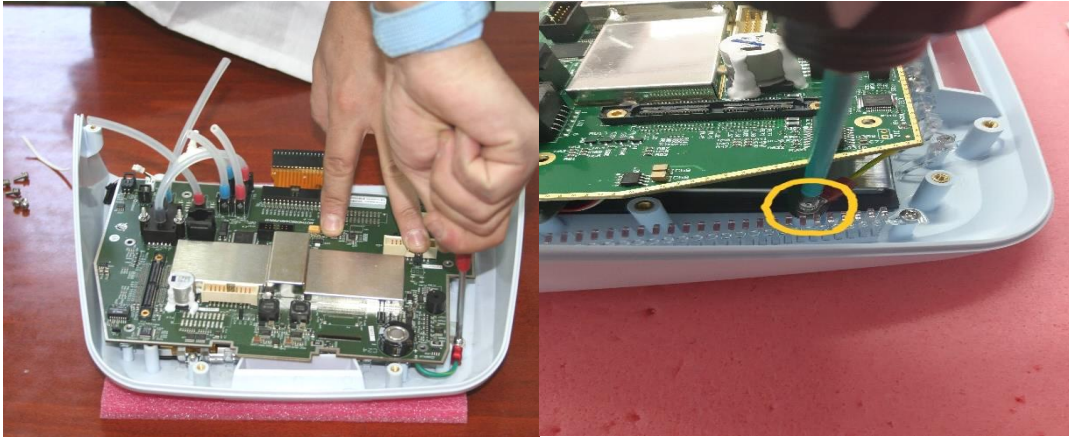

*Figure 29 – Detaching the Grounding Screw*

8. Lift the Main Board. Carefully detach the flat cables from the connectors on the main board.
10. Remove the Main Board.




Do not remove the silicon tubes from the main board when it is sent for repair.

### 4.10.2 Installing the Main Board

➔ To install the main board:

1. Connect the Ferrite to Flat cable

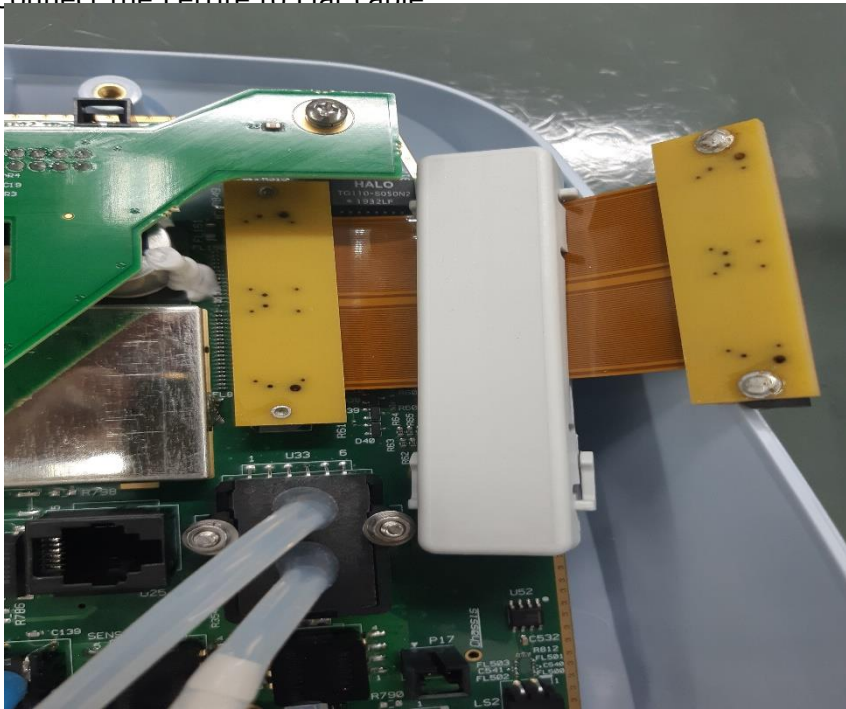

Check that the main board sockets do not have any bent pins.

2. Perform the removal procedure in the reverse order.( Remove the Ferrite from the cable by special key only).
3. Update “Total vent hours” with one time password – 1412.

## 4.11 Replacing the Touch Screen

The touch screen cannot be detached from the front panel; When the touch screen needs replacement , the Front Subassembly (P/N V60-70010-60) is to be replaced

1. Follow the procedures for removing the main board (see section 4.10)
  2. Replace the front Subassembly
- ➔ **To reassemble the touch screen:**
- Perform the above procedure in reverse order.

## 4.12 Replacing the Blower Assembly

### 4.12.1 Removing the Blower Assembly

➔ To remove the blower assembly:

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Disconnect the blower cable from the front panel assembly.
4. Disconnect the blower cable from the power board.
5. Disconnect the internal mixer tubing **\*(for IO2 models only – see Figure 30)**
6. Disconnect the Inlet and outlet silicon hoses (see Figure 30)



Figure 30 – Blower disconnection

7. Remove the 6 screws (3 on each side) securing the blower to the base – see Figure 31.

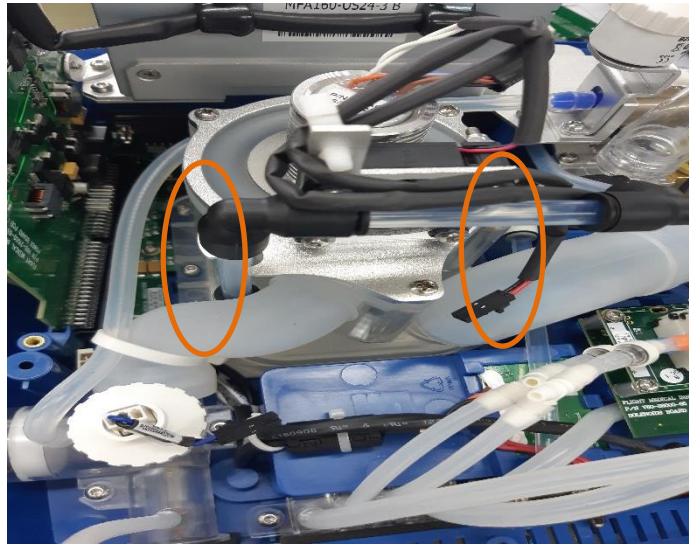


Figure 31- Blower screws

8. Remove the blower

#### 4.12.2 Installing the Blower Assembly

- ➔ To install a Blower assembly:
  - Perform the removal procedure in the reverse order.
  - Change Motor Hour meter value to "0"

### 4.13 Replacing the Oxygen Sensor



**Refer to Flight Medical Service Notes for a list of approved Oxygen sensors**

#### 4.13.1 Removing the Oxygen Sensor

- ➔ To remove the oxygen sensor:
  1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
  2. Remove the ventilator cover (see section 4.5).
  3. Remove the front panel (see section 4.8).
  4. Disconnect the electrical cable from the oxygen sensor.

Replacing the O<sub>2</sub> leak sensor \* (iO<sub>2</sub> Internal mixer model only)

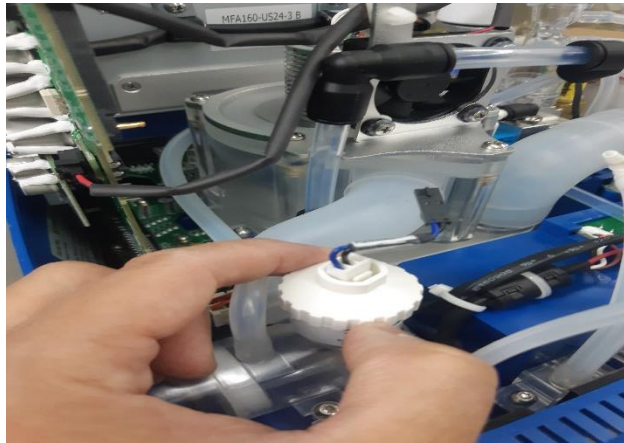


Figure 30 – Removing the Oxygen Sensor

5. Turn the oxygen sensor counterclockwise until it comes out.

### 4.13.2 Installing the Oxygen Sensor

- ➔ To install the oxygen sensor:
  - Follow the above procedure in reverse order.

## 4.14 Replacing the O<sub>2</sub> leak sensor \* (iO<sub>2</sub> Internal mixer model only)

### 4.14.1 Removing the O<sub>2</sub> leak sensor

- ➔ To remove the O<sub>2</sub> leak sensor:
  1. Remove the ventilator cover (see section 4.5).
  2. Disconnect the electrical cable from the O<sub>2</sub> leak sensor.
  3. Turn the O<sub>2</sub> leak sensor counterclockwise until it comes out.

### 4.14.2 Installing the O<sub>2</sub> leak sensor

- ➔ To install the oxygen sensor:
  - Follow the above procedure in reverse order.

## 4.15 Replacing the Oxygen mixer \* (iO<sub>2</sub> Internal mixer model only)

### 4.15.1 Removing the Oxygen mixer

➔ To remove the oxygen mixer:

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Remove the Power Supply and the power supply base plate + the solenoid (see section 4.7).
4. Remove the O<sub>2</sub> leak sensor
5. Remove the 2 silicon tubes and disconnect the Nebulizer & O<sub>2</sub> Mixer cables from the O<sub>2</sub> mixer Neb board
6. Gently pull out the O<sub>2</sub> mixer

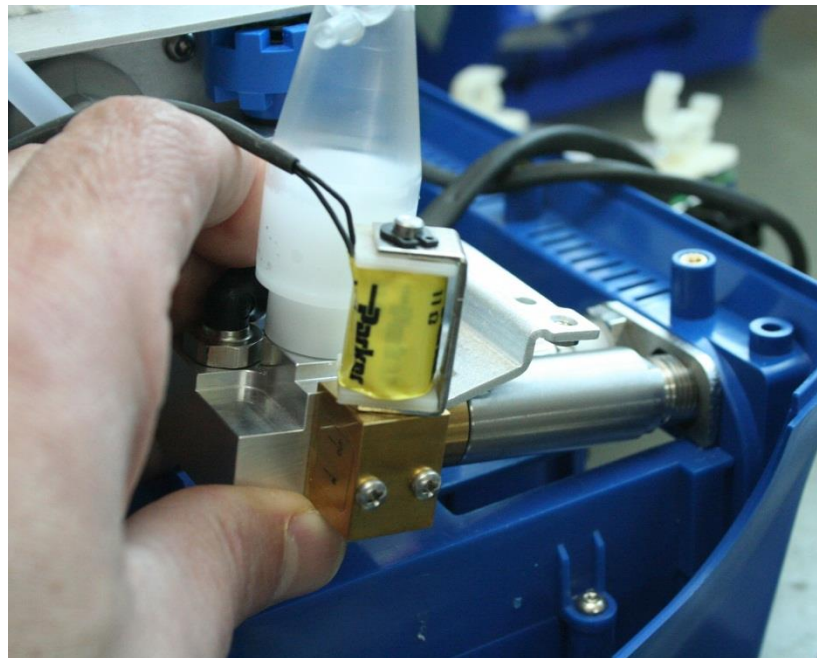


Figure 31 – Pulling out the mixer

### 4.15.2 Installing the Oxygen mixer

➔ To install the oxygen mixer:

- Follow the above procedure in reverse order. Make sure the flat surface of the mixer is aligned with the metal bracket.

Replacing the O<sub>2</sub> mixer filters \* (iO<sub>2</sub> Internal mixer model only)

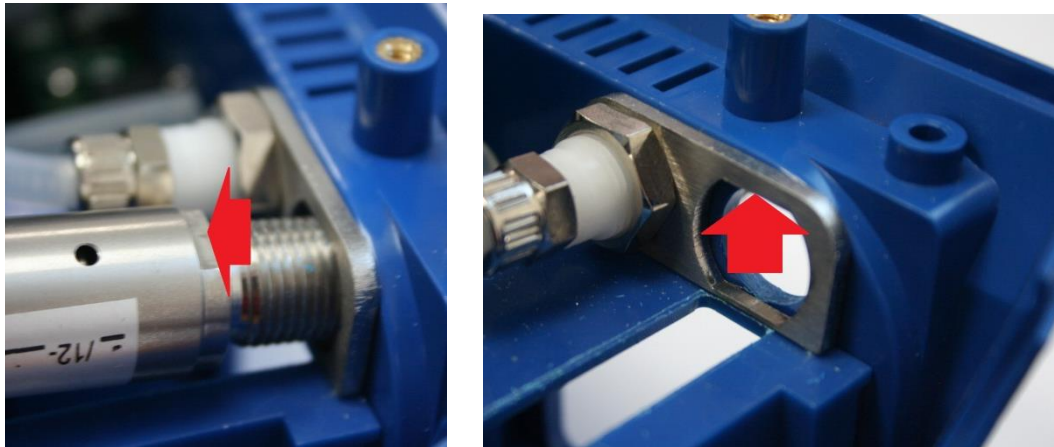


Figure 32 – Mixer orientation

#### 4.16 Replacing the O<sub>2</sub> mixer filters \* (iO<sub>2</sub> Internal mixer model only)

To replace the filters:

1. Use twizers to remove the O-ring that secures the filters
2. Use the twizers to pull out the filters (2 each)
3. Place the new filters (2 each) at the O<sub>2</sub> input and push all the way in
4. Push the O-ring in, verify it's in the groove, securing the filters in place

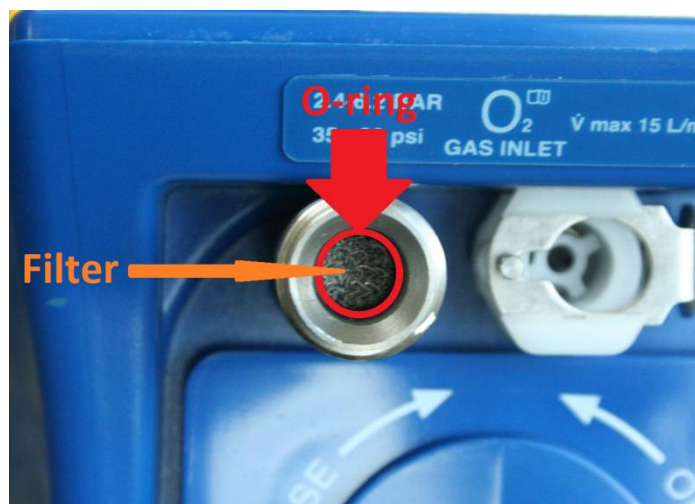



Figure 33 – O<sub>2</sub> regulator filter

## 4.17 Replacing the Purge Board Assembly

### 4.17.1 Removing the Purge Board Assembly

➔ To remove the purge board assembly:

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Remove the front panel (see section 4.8).
4. Disconnect the two check valves at the ends of the silicon tubes from the purge board.

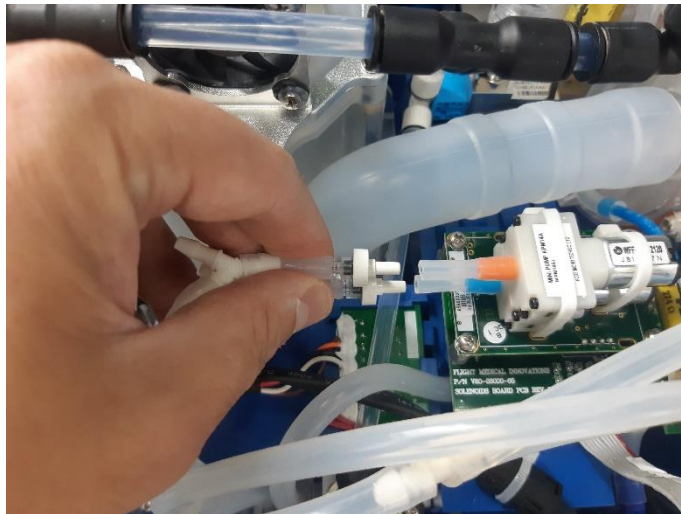

*Figure 34 – Disconnecting the Check Valves*

5. Remove the four Philips screws (star washers and flat washers) from the purge board. Carefully remove the stand-offs between the purge board and the solenoid board.
6. Carefully separate (pull out) the purge board from the connector on the solenoid board.

### 4.17.2 Installing the Purge Board Assembly

➔ To install the purge board assembly:

- Follow the above procedure in reverse order.

## 4.18 Replacing the Solenoid Board Assembly

### 4.18.1 Removing the Solenoid Board Assembly

- ➔ To remove the solenoid board assembly:
  1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
  2. Remove the ventilator cover (see section 4.5).
  3. Remove the front panel (see section 4.8).
  4. Remove the purge board (see section 4.14).
  5. Disconnect connector P1 and the flat cable from the solenoid board.


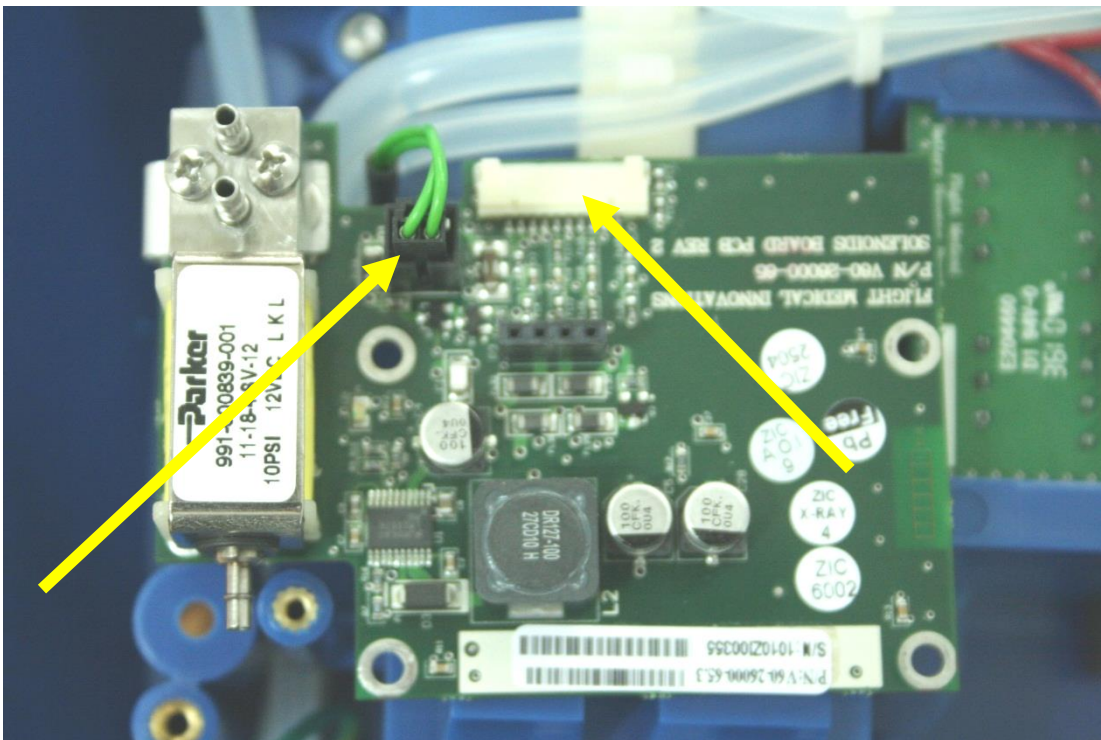


Figure 35 – Solenoid board connectors

6. Remove the silicon tubes from the fittings on the redundancy solenoid.

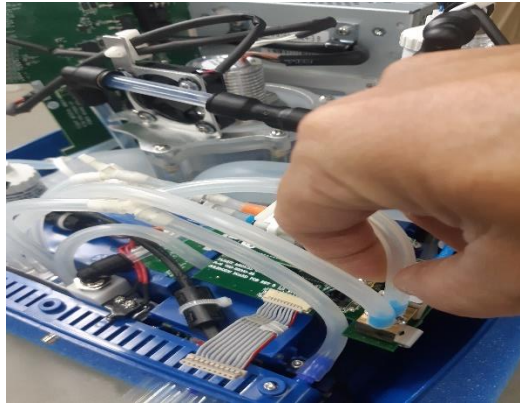


*Figure 36 – Removing the Silicon Tubes*

7. Remove the solenoid board assembly.

#### 4.18.2 Installing the Solenoid Board Assembly

➔ **To install the solenoid board assembly:**

1. Insert the solenoid assembly.
2. Insert the purge board assembly.
3. Connect the silicon tube attached to the female connector to the inner fitting on the redundancy solenoid.
4. Using the other silicon tube, connect the Solenoid assembly to the outer fitting on the redundancy solenoid.

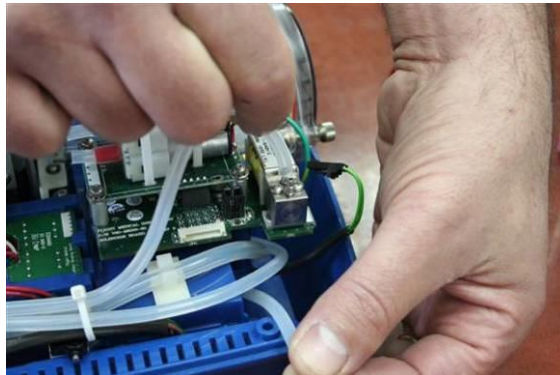

*Figure 37 – Connecting the Silicon Tubes*

## 4.19 Replacing the Solenoid Assembly

### 4.19.1 Removing the Solenoid Assembly

➔ To remove the solenoid assembly:

1. Remove the ventilator cover.

2. **For  $iO_2$  internal mixer model** - Remove the  $O_2$  leak sensor.

3. Remove the 2 screws securing the  $O_2$  leak sensor bracket. Remove the bracket.

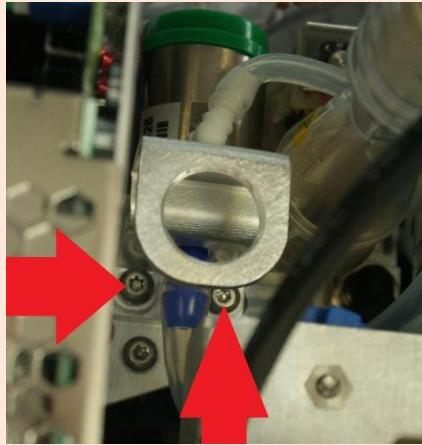


Figure 40 – Remove  $O_2$  leak sensor bracket

4. Disconnect the solenoid cable from the solenoid board (P1). If it's hard to reach you can remove the screws that secure the front panel and move the front panel so gain better access.
5. Disconnect the 3 silicon tubes from the solenoid assembly.
6. Unscrew the two adjacent Torx screws on the solenoid bracket. Pull out the bracket with the solenoid, carefully leading the solenoid cable, making sure it is not being damaged
7. Remove the 2 screws securing the solenoid to the bracket. Pull out the solenoid from the bracket. Verify the sealant O-ring is removed and is not attached to the bracket.

### 4.19.2 Installing the Solenoid Assembly

➔ To install the solenoid assembly:


- Follow the above procedure in reverse order.

## 4.20 Replacing the Lower Board and D-type Board

### 4.20.1 Removing the Lower Board and D-type Board

➔ **To remove the Lower and D-type boards:**

1. Remove the detachable battery (see section 4.3) and the internal battery (see section 4.4).
2. Remove the ventilator cover (see section 4.5).
3. Remove the front panel assembly (see section 4.8).
4. Remove the Power board (see section 4.6)
5. Remove the Power Supply and its base plate (see section 4.7)
6. Remove the internal O2 mixer (see section 4.15)
7. Remove the Blower assembly (see section 4.11)
9. Unscrew the 2 Torx screws (plastic washers) securing the Outlet assembly. Remove the Outlet assembly.

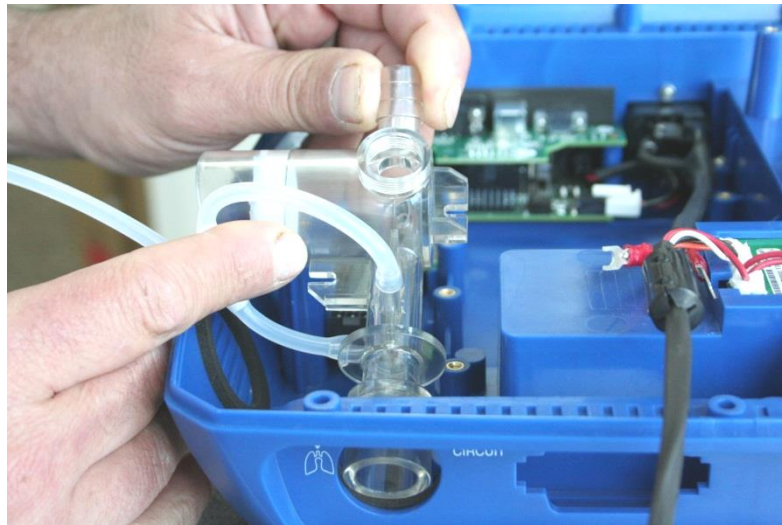

*Figure 41 – Removing the Outlet assembly*

10. Disconnect connectors P5, P6 and P7 from the lower board



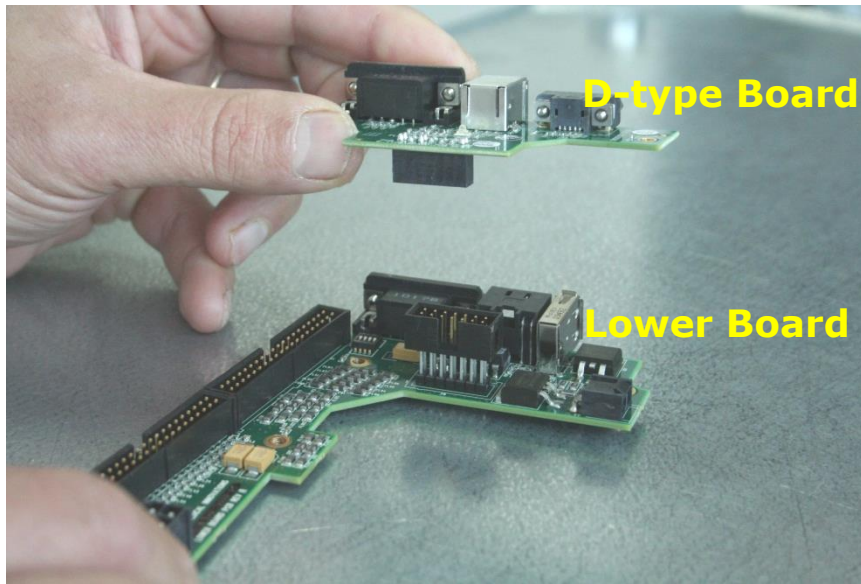


Figure 40 – D-type and Lower boards

## 4.20.2 Installing the Lower Board and D-type Board

- ➔ To install the Lower and D-type boards:
  - Follow the above procedure in reverse order.

## 4.21 Replacing the Fuse

The fuse does not need regularly-scheduled servicing. Servicing is required only in case of a problem.

### 4.21.1 Removing the Fuse

- ➔ **To remove the fuse:**
  1. Pull out the plastic fuse housing at the back of the device with the aid of a flat screwdriver.

- Installing the Fuse

- ➔ **To install the fuse:**

Reposition the fuse housing (flat edge on the left) and press firmly to make sure that it is inserted completely.

## Replacing the Rubber Bumpers

### 4.22 Replacing the Rubber Bumpers

The six bumpers on the underside of the device do not need regularly-scheduled servicing. Servicing is required only in case of a problem.

#### 4.22.1 Removing the Rubber Bumpers

➔ **To remove bumpers:**

1. Use a Torx screwdriver to remove bumpers as needed.

*Figure 41 – Removing a Rubber Bumper*



#### 4.22.2 Installing the Rubber Bumpers

➔ **To install a bumper:**

Use a Torx screwdriver to attach the bumper and verify that it is fastened securely.

## 5 Service/Technical Menu

### ➔ To enter the Service/Technical menu:

1. Press **Technical** / 
2. Press **SERVICE**.
3. Enter the code **5844**, and confirm by pressing 

### 5.1 Service Screen



Button	Options	Description
Enable B-LEVEL	ON/OFF	Enable/Disable B-LEVEL ventilation mode
Enable O2 mixer	O2 only O2+Neb. OFF	Nebulizer is disabled Mixer + Nebulizer
Enable HFOT	ON/OFF	Enable/Disable HFOT ventilation mode
Enable PRVC	ON/OFF	Enable/Disable PRVC breath control option
Enable Presets	ON/OFF	Enable/Disable quick start buttons for presets
Brightness	10 - 100 Resolution: 10	Increase/decrease the screen intensity
Language		Language options
Enable Sigh	ON/OFF	Enable/Disable Sigh option
Enable VG	ON/OFF	Enable/Disable VtG and MVG ventilation modes
Purge period	60 - 300 Resolution: 10	Set period between consequent purge operations in seconds
Saving Logs	ON/OFF	Activate log files download. Refer to Chap. 8
More		Open additional service screen

## 5.2 Additional service Screen




Button	Options	Description
Serial Port	OFF EGD JSON HL7	No communication via RS232 port Use RS232 port for communication with EGD Use RS232 port for JSON protocol Use RS232 port for communication with the communication box (KIT-0116)
SW Version		Displays the SW version of each CPU
Calibrate pressure		Activate pressure sensors calibration
Calibrate FiO2		Activate oxygen sensor calibration
Calibrate Bias Flow		Activate bias flow calibration
Calibrate Relief		Activate relief valve calibration
Configure system		<ul style="list-style-type: none"> <li>- Change serial numbers;</li> <li>- Enter the code <b>2009</b>, confirm by pressing ✓ ;</li> <li>- Change Motor Hour Meter enter the single time code <b>1412</b>, confirm by pressing ✓</li> </ul>
Factory defaults		Restore default calibration parameters and factory configuration of parameters. Enter code <b>2009</b> , confirm by pressing ✓
Neb. Comp. *	0 – 10.0 Resolution: 0.1	Volume compensation when nebulizer is on.
Calibrate O2 Leak *	10 - 100	Activates oxygen leak sensor calibration

	Resolution: 10	
Bar Intensity	Single/Full	Number of lit LEDs in the ventilator's pressure gauge
More		Opens previous service screen/HL7 configuration screen if HL7 serial port is chosen

**\*  $iO_2$  INTERNAL MIXER MODEL ONLY**

## 5.3 HL7 Configuration Screen

For more detailed description and instructions see: DOC-0976 A00 Flight 60T HL7 Interface Guide.

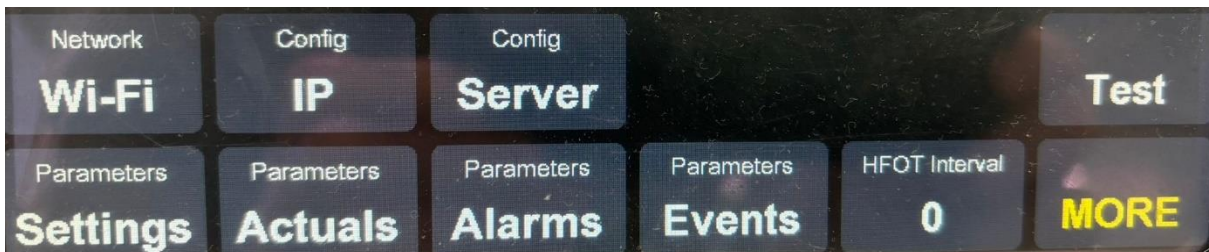

### 5.3.1 Network

The network button contains two options.

#### 5.3.1.1 Ethernet

Sending messages over Ethernet cable (RJ-45).

#### 5.3.1.2 WiFi

Sending messages over Wi-Fi protocol (IEEE 802.11).

In this option the antenna must be plugged to the W610 communication box.

### 5.3.2 Config IP

#### 5.3.2.1 Ethernet connection screen

Choose "Manual" to manually configure the IP address by filling the following fields:

- a. IP
- b. Mask
- c. Gateway

Choose "Auto" to automatically receive an IP address.

#### 5.3.2.2 WiFi connection screen

Press the "Scan" button to start scanning for available networks. Use the up/down

## HL7 Configuration Screen

keypad buttons to scroll between them.

Press "Password" to open a T9 keyboard to enter the selected network password.

### 5.3.2.3 Apply

Once configuration is done press "Apply" to start the connection process. This may take up to 1 min.

Status message will appear next to the "Apply" button as follows:

- Connecting – during the connection process
- Connected – successful connection
- Fail – connection process failed (time out)

## 5.3.3 Config Server

Make sure valid network connection before server configuration.

### 5.3.3.1 HL7 IP

Set here the HL7 server address. Each Octata can hold number between 0-255

### 5.3.3.2 HL7 Port

Set here the HL7 Port. There is placeholder for 5-digit number (range 0-99999)

### 5.3.3.3 Server Ack

#### 5.3.3.3.1 *ON*

Acknowledge messages are received from the server.

A green "HL7" connectivity indicator will be shown on the main screen. If no acknowledge message is received in 20 sec, the HL7 indicator will turn red.

#### 5.3.3.3.2 *OFF*

No acknowledge messages are received from the server.

In this case there will be no indicator for connectivity on the main screen.

### 5.3.3.4 Apply

Once configuration is done press "Apply" to start the connection process. This may take up to 1 min.

If Server "Ack" is set to ON, a dummy message is sent to the server and status message will appear next to the "Apply" button as follows:

Connecting – during the connection process

Connected – successful connection

Fail – connection process failed (time out)

Invalid port – port no. is not in the allowed range

Invalid IP – port IP no. is invalid

No network connection – no internet connection

### **5.3.4 Test**

Pressing "Test" button will simulate message sending to the server.

### **5.3.5 Parameters**

The parameters being sent to the server can be configured by the user by checking the box next to each parameter. Only parameters that are relevant to the specific mode/status will be sent.

## 6 Software Upgrade

### Required equipment:

- USB storage device with the SW version (*.pkg* file)



Software upgrade will not run if the batteries are charged below 50%.

### 6.1 Installing the SW upgrade

1. Turn off the ventilator, disconnect from AC.
2. Copy the new SW version (*.pkg* file) to a USB storage device.
3. Plug the USB storage device to the vent USB port.
4. Connect the ventilator to AC.
5. The ventilator powers up and a software update screen opens, displaying the current version and the new version to update.
6. Press **OK** on the touch screen.
7. Once upgrade is done, the ventilator displays " Finished updating, please remove DOK"
8. Remove the USB storage device and press **OK** on the touch screen.
9. Turn on the ventilator, press **Technical** /  and press **Show Info**.
10. Verify the new software version is displayed.

### 6.2 Calibration integrity check

1. Set the ventilator to the same ventilation modes:
  - ACMV
  - VT = 500 mL
  - PEEP = 5
  - P trig = -5.0
  - Rate = 15
  - Ti = 1
  - %O<sub>2</sub> = 80%
  - Waveform = Square
2. Connect to a patient circuit. Perform circuit test.
3. Start ventilation.
4. Verify the following:

**V<sub>ti</sub>**            Within the 450-550 spec.

**P base**         $5 \pm 1$ cm H<sub>2</sub>O

**FiO<sub>2</sub>**                76% - 84% for Internal O<sub>2</sub> mixer (iO<sub>2</sub>) model

5. If any of those values is out of spec, perform the appropriate calibration and check again.

## Required Equipment

# 7 Calibration and Verification tests




## Required Equipment


- Adult test lung specified up to 90cm H<sub>2</sub>O with adjustable lung compliance of C=10;20;50
- Pressure/flow analyzer, specified to 120cm H<sub>2</sub>O, accuracy of 0.1 cmH<sub>2</sub>O.
- Screwdrivers:
  - Phillips screwdriver PH1
  - Flat screwdriver 0.8x4
  - Torx screwdriver BT10x200
- Reusable patient circuit (P/N V60-50000-60)
- Vacuum gauge, range to -1bar
- Oxygen Analyzer
- Parabolic resistors: Rp5;Rp20;Rp50 Made by Michigan Instruments Inc.
- Medical-grade Oxygen source (pressure over 50PSI)
- \* Oxygen Mixer (By Flight Medical, P/N V13-00010-60) – for non IO2 models.

## 7.1 Calibration

### 7.1.1 Pressure Sensors Calibration

➔ To calibrate the pressure sensors:

1. Connect the ventilator to an external pressure meter using a reusable patient circuit.
2. Connect the pressure meter to the test lung.
3. Use the following settings for the test lung:
  - C = 10
  - Rp = 5
4. Press **Technical** /  and then press **SERVICE**.
5. Enter the code **5844**, and confirm by pressing .
6. Press **MORE**.
7. Disconnect the patient circuit from the ventilator to verify there is no pressure buildup in the circuit. Reconnect the circuit to the ventilator
8. Press **Calibrate Pressure** and then **Enter** / . The ventilator starts to build pressure.
9. The following screen opens:

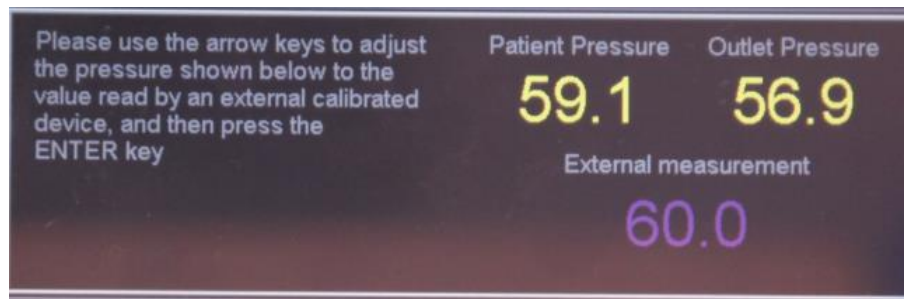


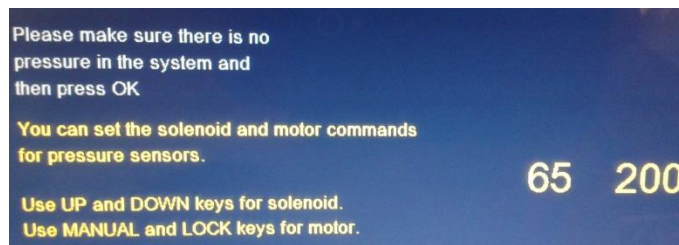
10. Check the **Patient Pressure** and **Outlet Pressure** values, verify the following:
- Values match the pressure measurement of the external pressure meter.
  - Values are between 45 and 75 cmH<sub>2</sub>O.
  - The difference between the **Patient Pressure** and **Outlet Pressure** values is no more than 0.1.


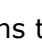






**If one of the conditions above is not met, calibrate the pressure sensors:**

1. **If the pressure on the external pressure meter is in the 50-75 cmH<sub>2</sub>O range**, continue to step 2.



If the pressure is less than 50 cmH<sub>2</sub>O:

- Press **Cancel** / .
- Press **Calibrate Pressure** again. The calibration screen will come up with the following message " *You can set the solenoid and motor commands for pressure sensor*" and displays default values of **65 200**.



- Use the **UP** /  and **DOWN** /  buttons to change the solenoid command (initial value of 65) or **Manual Breath** /  and **Panel lock** /  buttons to change motor command (initial value of 200) and press **Enter** / .
  - Verify that the pressure on the external pressure meter is in the 50-75 cmH<sub>2</sub>O range. If not then repeat the above steps until the pressure is the correct range.
  - Continue to step 2.
2. Use the **UP** /  and **DOWN** /  buttons to change the **External measurement** value of 60 to the actual pressure reading of the external pressure meter.
  3. Press **Enter** / . The ventilator stops.

### Calibration




4. Press **Calibrate Pressure** again and then **Enter** / .
5. Check the **Patient Pressure** and **Outlet Pressure** values, verify the following:
  - Values match the pressure measurement of the external pressure meter.
  - Values are between 50 and 75 cmH<sub>2</sub>O.
  - The difference between the **Patient Pressure** and **Outlet Pressure** values is no more than 0.1.
6. Press **Cancel** /  to exit.

### If the test fails:

1. Replace patient circuit and redo calibration.
2. Do one of the following:
  - If the external measurement lower than 50:
    - Check internal tubing connections for air leaks.
    - Replace the solenoid and redo calibration.
  - If the external measurement higher than 75:
    - Check internal tubing connections for blocking.
    - Replace the solenoid and redo calibration.
  - If the patient pressure / outlet pressure difference is more than 0.1cmH<sub>2</sub>O:
    - Repeat the calibration up to 3 times. If the desired results are not obtained after three attempts:
      - Examine the silicon tube connections and try to locate a leak.
      - Verify that the relief valve is not activated.
      - Replace the solenoid (see 4.19). Perform the required calibrations (see 7.3 Test table).
      - Replace the Main Board (see 4.10). Perform the required calibrations (see 7.3 Test table).

## 7.1.2 Pressure Relief Calibration

➔ **To calibrate the pressure-relief mechanism:**

1. Connect the ventilator to the test lung using a patient circuit
2. Set the test lung to minimum compliance
3. Press **Technical** /  and then press **SERVICE**.
4. Enter the code **5844**, and confirm by pressing 
5. Press **MORE**.
6. Press **Calibrate Relief** and then **Enter** /  to start the motor.
7. Verify that there is no air leakage from the exhalation valve. Air should only be coming out via the pressure-relief valve.
8. Verify that the displayed pressure reading is between 105 and 120 cmH<sub>2</sub>O.
9. If the pressure is not within this range, adjust the Relief valve:
  - While the calibration is running, place the ventilator in a way that the calibration screw is accessible from below (Refer to picture *Figure 42 – Adjusting the Pressure Relief Valve* below).
  - Use a Philips screwdriver to adjust the relief valve by tightening or loosening the calibration screw located on the underside of the outlet fitting.



*Figure 42 – Adjusting the Pressure Relief Valve*

10. Press **Enter** /  to confirm the reading and to complete the calibration.



If the calibration screw on the pressure-relief valve was adjusted, repeat the calibration of the pressure sensors.

**If the test fails:**

- Check internal tubing connections.

Calibration

### 7.1.3 Bias Flow Calibration




➔ To calibrate the Bias Flow

1. Connect the circuit to the ventilator.
2. Remove the flow sensor from the end of the patient circuit. **Connect a test lung to the patient circuit.**
  - On an SL ventilator, connect the flow sensor to the exhalation valve as per the following picture.



- On a DL ventilator, connect the flow sensor to the DL exhalation valve.



3. Press **Technical** /  and then press **SERVICE**.
4. Enter the code **5844**, and confirm by pressing .
5. Press **MORE**.
6. Press **Calibrate Bias Flow** and then press the **Manual Breath** / . Wait for the displayed value to stabilize at 15.
7. Press **Enter** / .
8. Repeat the Bias Flow calibration (steps 5-6) once more.
9.
  - When using a D.L ventilator with a single patient circuit, Bias Flow calibration with a single patient circuit must be perform.





**If the test fails:**

1. Check internal tubing connections for air leaks.
2. Run the following tests:
  - Pressure sensor calibration
  - Circuit test
  - Pressure relief calibration
3. If fails the test again :
  - Please contact support in order to Replace the solenoid .

### 7.1.4 Oxygen Sensor Calibration

➔ To calibrate the Oxygen sensor:





***For iO<sub>2</sub> internal mixer model***

1. Connect a high pressure oxygen hose to the High Pressure O<sub>2</sub> port
2. Connect a calibrated Oxygen Analyzer to the ventilator's outlet.
3. **Connect a patient circuit.**
4. Press **Technical** /  and then press **SERVICE**.
5. Enter the code **5844**, and confirm by pressing .
6. Press **MORE**.
7. Press **Calibrate FiO<sub>2</sub>**.
8. Choose **USED** if calibrating the current sensor, **NEW** if O<sub>2</sub> sensor is replaced.
9. The ventilator performs 21% calibration. Verify the external Oxygen analyzer reads 21%, wait 1 minute and press **Enter** / .
10. The ventilator switches to 100% automatically. Verify the external Oxygen analyzer reads 100%, wait 1 minute and press **Enter** / .

***For External mixer model***






1. Connect the Air/Oxygen Entrainment Mixer to an Oxygen source. Install the Air/Oxygen Entrainment Mixer on the ventilator's air inlet.
2. Set the regulator on the oxygen source to 50 PSI.
3. Connect a calibrated Oxygen Analyzer to the ventilator's outlet.
4. **connect a patient circuit**

### Calibration

5. Press **Technical** /  and then press **SERVICE**.
6. Enter the code **5844**, and confirm by pressing .
7. Press **MORE**.
8. Press **Calibrate FiO2**.
9. Choose **USED** if calibrating the current sensor, **NEW** if O<sub>2</sub> sensor is replaced.
10. Set the Entrainment Mixer dial to AIR.
11. Once the external Oxygen sensor displays 21%, wait 1 minute and then press **Enter** / .
12. Set the Entrainment Mixer dial to 100%.
13. Once the external Oxygen sensor displays 100%, wait 1 minute and then press **Enter** / .

### 7.1.5 O<sub>2</sub> Leak Sensor calibration\* (*iO<sub>2</sub> Internal mixer model only*)

➔ **To calibrate the O<sub>2</sub> leak sensor:**

1. Disconnect the ventilator from any Oxygen source.
2. Connect a calibrated Oxygen Analyzer to the ventilator's outlet. **Don't connect a patient circuit!**
3. Press **Technical** /  and then press **SERVICE**.
4. Enter the code **5844**, and confirm by pressing .
5. Press **MORE**.
6. Press **Calibrate O2 Leak**. The system will be flushed with ambient air.
7. Wait 1 minute. Use the **UP** /  and **DOWN** /  buttons to match the displayed value with the external Oxygen analyzer reading.
8. Press **Enter** / .

## 7.2 Operational Verification Procedure (OVP)

### 7.2.1 Standard Ventilator Settings (STS)

Refer to the following specifications whenever *STS settings* are called for:

- ACMV
- VT = 500 mL
- PEEP = 0
- P trig = -1.5
- Rate = 15
- Ti = 1
- Waveform = Descend


**For test lung:**

- Rp=5
- C = 50 mL / cmH<sub>2</sub>O

**For IMT/CITREX:** – ATP SET

### 7.2.2 Front Panel Self Test

➔ **To run the self test:**

1. Connect the ventilator to AC power.
2. Press the **On Off** /  button on the front panel.
3. During the self test that begins automatically, check for the following:
  - Short audible beep
  - All LEDs on the **LED indicator** light up with the same intensity, with the following colors:

Description	Colour
TRIG (Pressure trigger)	Green
EXT PWR (External power)	Yellow
LOW BAT (Low battery)	Red
BAT (Battery power)	Orange
FAULT (Device malfunction)	Red
APNEA	Red
BUV (Back up ventilation)	Red

**Operational Verification Procedure (OVP)**

<b>HIGH MV</b> (High minute ventilation)	Red
<b>LOW MV</b> (Low minute ventilation)	Red
<b>High P</b> (High pressure)	Red
<b>LOW P</b> (Low pressure)	Red
<b>Audio Paused/Alarm Reset</b>	Orange
<b>On/Off</b>	Green
<b>Panel Lock</b>	Orange

- All LEDs between 60 – 100 on the **pressure gauge** light up.

**If the test fails:**


**No LEDs are on** - Check the internal connectors to the front panel.


**Some LEDs are on** - Replace the Main Board


### 7.2.3 Circuit Test

Each time the Exhalation Valve or patient circuit is replaced or serviced, it must be tested.

➔ **To test the patient circuit / exhalation valve:**

1. Connect the patient circuit to a 0.5L test lung.
2. Press **Technical** / , and then press **Circuit TEST**.

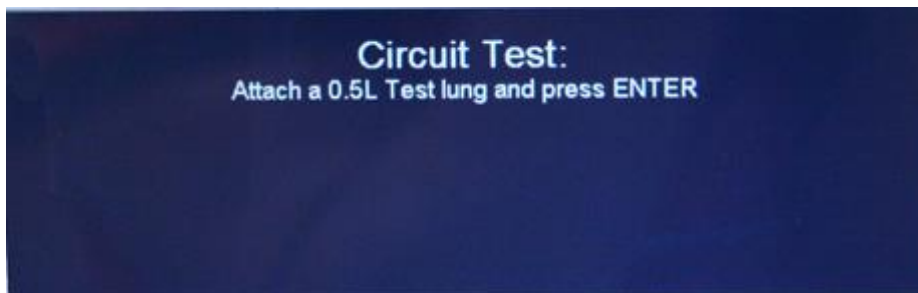

3. Press **Enter** /  .
4. The test is done automatically. Checking 3 PEEP points – 20; 10; 5
5. If the test is successful, the display shows a **Test PASSED** message

**If the test fails:**

1. Press **Cancel** / **⊗**.
2. Replace the patient circuit. In case of a Flight60 DL check the exhalation valve diaphragm.
3. Perform the calibration again.
4. Check internal tubing connections for air leaks.
5. Replace the solenoid and perform the required calibrations (see 7.3 Test table).

## 7.2.4 Volume verification

➔ **To test the device for pressure:**

1. Set the ventilator to STS settings - PEEP=5.
2. Activate the ventilator and verify that the volume is between 425 and 575ml.

**If the test fails:**

1. Replace patient circuit
2. Replace test lung
3. Check\Replace Exhalation Valve
4. Check\Replace Exhalation (blue) diaphragm
5. Repeat the Pressure calibration

## 7.2.5 Pressure Verification

➔ **To test the device for pressure:**

1. Set the ventilator to STS settings.
2. Connect a pressure meter between the ventilator and the test lung.
3. Use the following settings for the test lung:
  - C = 50
  - Rp = 5
4. Activate ventilation.
5. Set  $T_i = 2$ .

### Operational Verification Procedure (OVP)

6. Verify that the external pressure meter reading and the **PIP** value displayed on the ventilator differ by no more than 2 cmH<sub>2</sub>O or 10%. (The higher of the two can be used for this determination.)

#### If the test fails:

Repeat the following:

- Pressure sensors calibration
- Pressure relief calibration
- Circuit Test

## 7.2.6 Bias Flow

### ➔ To test the Bias Flow:

1. Set the test lung to compliance of 20ml/cmH<sub>2</sub>O, resistance 20
2. Set the ventilator to STS, **P trig** = -3; **Rate** = 8; **PEEP** = 5
3. Connect a Flow meter between the patient circuit and the exhalation valve.

Start ventilation:

- Verify that the Flow meter reading is between 10-20 L/m

Set **PEEP** = 10

4. Verify that the Flow meter reading is between 10-20 L/m

#### If the test fails:

1. Perform Bias Flow calibration

## 7.2.7 PEEP

### ➔ To test the PEEP:

1. Use the following settings for the test lung:
  - C = 50
  - R<sub>p</sub> = 5
  - If a test lung with analysis software is used, configure the analysis software program to the same settings
2. Connect a pressure meter between the ventilator and the test lung.

3. Set the ventilator to STS settings.
  4. Set **PEEP** to 5.
  5. Activate ventilation.
  6. After 5 breaths, check that the basic pressure levels in the lung and in the ventilator display are  $5 \pm 1$  cmH<sub>2</sub>O.
  7. Raise the **P trig** level as needed, until no auto-triggering is observed.
  8. Verify that the **P trig** level is not more than -1.5 cmH<sub>2</sub>O, and that the ventilator is not performing an auto-trigger.
- 
9. Set **PEEP** to 10.
  10. After five breaths, check that the basic pressure levels in the lung and in the ventilator display are  $10 \pm 2$  cmH<sub>2</sub>O.
  11. Raise the **P trig** level as needed, until no auto-triggering is observed.
  12. Verify that the **P trig** level is not more than -2 cmH<sub>2</sub>O.
- 
13. Set **PEEP** to 20.
  14. After five breaths, check that the basic pressure levels in the lung and in the ventilator display are  $20 \pm 2$  cmH<sub>2</sub>O.
  15. Raise the **P trig** level as needed, until no auto-triggering is observed.
  16. Verify that the **P trig** level is not more than -3 cmH<sub>2</sub>O.

**If the test fails:**

1. Replace the patient circuit. In case of a Flight60 DL replace the exhalation valve diaphragm.
2. Repeat Circuit test.
3. Replace the solenoid and run the required calibrations (see 7.3 Test table).

## 7.2.8 Pressure Trigger

**➔ To test the pressure trigger:**

1. Set the ventilator to STS
2. Connect the ventilator to the test lung using a patient circuit. Use the following settings for the test lung:
  - C = 50
  - Rp = 5
3. Activate ventilation.
4. Simulate inhalation attempt by creating a negative pressure from the test lung.

### Operational Verification Procedure (OVP)

5. Verify that the green **P trig** LED flashes and a mandatory breath is forced each time the edge (shoulder) is raised.
6. Change settings to:
  - **SIMV**
  - **PS above peep** = 20
  - **Rate** = 5.
7. Simulate inhalation attempt by creating a negative pressure from the test lung.
8. Verify that the green **P trig** LED flashes and that the ventilator provides a mandatory breath followed by pressure-supporting ventilations (PSV) for 12 seconds.
9. Set the **P trig** level to -0.5 cmH<sub>2</sub>O, and verify that no auto-triggering occurs.

#### If the test fails:

1. Replace the patient circuit. In case of a Flight60 DL replace the exhalation valve diaphragm.
2. Repeat the following:
  - Pressure sensors calibration
  - Pressure relief calibration
  - Circuit test
3. Replace the solenoid and perform the required calibrations (see 7.3 Test table).

## 7.2.9 Exhalation Valve Leak

#### ➔ To test the exhalation valve:

1. Set the ventilator to the following settings:
  - **Rate** = 10
  - **Ti** = 1.4
  - **ACMV**
  - **VT** = 1400
2. Set the High-Pressure alarm to 99 cmH<sub>2</sub>O.
3. Connect the ventilator to a test lung using a patient circuit. Use the following settings for the lung:
  - **C** = 20
  - **Rp** = 20


4. Activate ventilation and continue for 15 breaths.
5. Verify that the pressure level is over 72 cmH<sub>2</sub>O. If pressure doesn't get to 72 cmH<sub>2</sub>O, decrease the compliance or increase the resistance of the test lung. If pressure becomes too high after changing compliance/resistance, reduce the set volume from 1400 until achieving the required pressure level.
6. Check for air flow from the underside of the Exhalation Valve during inhalation; If air flow is noticed there is a leak and the test fails.

**If the test fails:**

1. Replace the patient circuit. In case of a Flight60 DL replace the exhalation valve diaphragm.
2. Repeat the following:
  - Pressure sensors calibration
  - Pressure relief calibration
  - Circuit test
3. Replace the solenoid (see 4.19), and perform the required calibrations (see 7.3 Test table)

## 7.2.10 High Pressure Alarm

**→ To test the high-pressure alarm:**

1. Set the ventilator to STS settings
2. Connect the ventilator to a test lung using a patient circuit.
3. Activate ventilation.
4. Set **HIGH P** to 10 cmH<sub>2</sub>O.
5. Check that when the pressure reaches 10 cmH<sub>2</sub>O, the following occurs:
  - the audible alarm is triggered
  - the visual alarm is triggered
  - inhalation is interrupted.
6. Set **HIGH P** to 99 cmH<sub>2</sub>O.
7. Check that the following occurs:
  - the audible alarm is silenced automatically
  - the visual alarm is displayed until the **Alarm Reset** /  button is pressed.

---

Operational Verification Procedure (OVP)

**If the test fails:**

1. Replace the Main Board (see 4.10).
2. Perform the required calibrations (see 7.3 Test table).

## 7.2.11 Battery /Charger /Power Supply

➔ **To test the power supply:**





Verify that both batteries are above 60%. If not, connect to external power and charge for at least 3 hours.

1. Ventilate is STS mode.
2. Disconnect the ventilator from AC power
3. Verify that the following occur:
  - the **Power Switchover** indicator is displayed
  - an audible beep is heard
  - the **EXT PWR** indicator is not lit (external power)
  - the battery (**BAT**) LED indicator is lit
4. Verify that the ventilator functions normally with no further alarms.
5. Remove the Main battery. Verify that the vent switched to Low-Power-Ventilation:  
**PEEP = 5, Ptrig = -20 , PC above PEEP = 15**
4. After at least 90 seconds have passed, reconnect to AC power.
5. Verify that within 30 seconds of the connection time the ventilator indicates switching back to AC power.
6. Verify that the following occur:
  - the battery (**BAT**) LED indicator is off
  - the green **EXT PWR** is lit

## 7.2.12 Buzzer Test

➔ To test the buzzer:

Press **Parameters** /  and choose **ALARMS**.

3. Tap the **Buzzer** button.
4. Choose HIGH and press **Enter** / . Verify that a High buzzer sound is heard.
5. Tap the **Buzzer** button.
6. Choose LOW and press **Enter** / . Verify that a Low buzzer sound is heard.

**If the test fails:**


- Check the buzzers cables. Make sure they are connected to the Main Board and are intact.
- Replace the Main Board (see 4.10), and perform the required calibrations (see 7.3 Test table).

## 7.2.13 Oxygen Calibration Test

### For iO<sub>2</sub> ventilator (with internal O<sub>2</sub> mixer)

Connect the ventilator to an Oxygen source.

O<sub>2</sub>% settings are changed by the %O<sub>2</sub> button on the touch screen

1. Press **Technical** /  and then press **O<sub>2</sub> sensor**.
2. Select **ON**.
3. Set the Low FiO<sub>2</sub> and the High FiO<sub>2</sub> alarm to **OFF**.
4. Connect the patient circuit via a calibrated Oxygen Analyzer to the test lung.
5. Set the ventilator to STS mode. Set PEEP=5. Start ventilation.
6. Change O<sub>2</sub> settings according to the following test table:

Input pressure	Mixer settings	Tolerance for IO <sub>2</sub> internal mixer
<b>50 PSI</b>	40%	36% - 44%

**Operational Verification Procedure (OVP)**

	60%	57% - 63%
	80%	76% - 84%
	100%	95% - 100%

**If the test fails:**





1. Check for an inlet leak.
2. Replace the sensor cable.
3. Replace the sensor.
4. Replace the Main Board (see 4.10) and perform the required calibrations (see 7.3 Test table).

**7.2.14 Nebulizer compensation test \* (*iO<sub>2</sub> Internal mixer model only*)**

➔ **To test the Nebulizer compensation**

1. Set the ventilator to STS mode.
2. Connect a nebulizer circuit **before the patient circuit's flow sensor**. Start ventilation.
3. Verify that Vti is 450-550.
4. Set O<sub>2</sub> mixer to 50%. Activate the Nebulizer.
5. Verify that Vti remains in the 450-550 range.

**If the test fails:**

1. Press **Technical** /  and then press **SERVICE**.
2. Enter the code **5844**, and confirm by pressing .
3. Press **MORE**.
4. Use the **UP** /  and **DOWN** /  buttons to change the **Neb. Comp.** value. Increasing the value will decrease Vti.

## 7.2.15 OVP Test Results

Operational Verification Procedure (OVP)

<b>Serial #:</b>	<b>SW Version:</b>	<b>Hours:</b>
<b>Model: Flight 60T</b>	<b>Tested By:</b>	<b>Date:</b>

<i>Test</i>	<i>Results</i>	<i>Pass</i>
<i>Front Panel self- test</i>	-N/A-	<input type="checkbox"/>
<i>Pressure Sensors Calibration</i>	External pressure gauge ..... Patient Pressure ..... Outlet Pressure .....	<input type="checkbox"/>
<i>Pressure Relief Valve Calibration</i>	Relief Pressure .....	<input type="checkbox"/>
<i>Bias Flow Calibration</i>	-N/A-	<input type="checkbox"/>
<i>Circuit Test</i>	-N/A-	<input type="checkbox"/>
<i>Volume Verification</i>	Vti ..... Vte .....	<input type="checkbox"/>
<i>Pressures Verification</i>	External gauge peak Pressure ..... PPeak .....	<input type="checkbox"/>
<i>Bias Flow test</i>	PEEP = 5 Bias Flow 10-20 L/min.	<input type="checkbox"/>
	PEEP = 10 Bias Flow 10-20 L/min.	<input type="checkbox"/>
<i>PEEP</i>	<b>Set</b> <b>Tolerance</b> <b>PEEP Value</b> <b>Ptrig</b>	<input type="checkbox"/>
	5            5 ± 1	
	10           10 ± 2	
	20           20 ± 2	
<i>Pressure Trigger test (P trig)</i>	-N/A-	<input type="checkbox"/>
<i>Exhalation Valve leak</i>	Pressure level .....	<input type="checkbox"/>
<i>High Pressure Alarm</i>	-N/A-	<input type="checkbox"/>
<i>Power Supply and Battery Charge</i>	-N/A-	<input type="checkbox"/>
<i>Alarm Buzzer Test</i>	-N/A-	<input type="checkbox"/>
<i>O2 Sensor</i>	<b>Set</b> <b>Tolerance</b> <b>Value</b>	<input type="checkbox"/>
	Air              20.9% - 29%	
	40%             32% - 48%	
	60%             52% - 68%	
	80%             72% - 88%	
	100%            92% - 100%	

### 7.3 Test Table

The following table summarizes tests associated with each device module. When a module is replaced, All tests associated with the replacement must be performed before the ventilator is used again.

Module	Detachable battery	Internal battery	Power board	Power supply	Front panel	Main board	Mixer board	Manifold	O <sub>2</sub> sensor	O <sub>2</sub> leak sensor	Purge board	Solenoid board	Solenoid	Internal O <sub>2</sub> mixer	Fuse
Calibration /Test															
Pressure sensor calibration					✓	✓		✓				✓	✓		
Pressure relief calibration						✓									
Volume factor calibration					✓	✓		✓							
Oxygen sensor calibration					✓	✓			✓						
O <sub>2</sub> leak sensor calibration							✓			✓					
Pressure verification			✓	✓	✓	✓		✓	✓		✓	✓	✓		
PEEP					✓	✓		✓			✓	✓	✓		
Pressure trigger					✓	✓		✓			✓	✓	✓		
Expiration Valve leak								✓	✓		✓	✓	✓		
High pressure alarm					✓	✓									
Battery/charger/power supply	✓	✓	✓	✓	✓	✓									✓
Inlet leak								✓							
Buzzer test	✓	✓	✓	✓	✓	✓									
Internal O <sub>2</sub> mixer							✓		✓					✓	
Nebulizer							✓		✓					✓	
Front panel			✓	✓	✓	✓									✓






## 8 Maintenance

### 8.1 Scheduled maintenance

**BLOWER** – The Blower should be replaced every 25,000 hours of operation or 5 years, whichever comes first, an alarm will go on when replace due time ""pump service due"

For instructions on replacing the Blower, see section 4.12

After replacing the Blower, update the Blower serial number and zero the working hours:

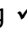1. Press **Technical** /  and then press **SERVICE**.
2. Enter the code **5844**, and confirm by pressing 
3. Press **MORE**.
4. Press "**Configure SYSTEM**".
5. Enter the code **2009**, and confirm by pressing 
6. Press the displayed **Compressor Serial Number**, use the numerical keyboard to type in the new number.
7. Press the displayed **Motor Hour Meter**, use the numerical keyboard to change it to "0".

**AIR INLET FILTER** – Check the filter's condition whenever servicing the ventilator.



**OXYGEN (O<sub>2</sub>) SENSOR** – Replace once a year.

**O<sub>2</sub> LEAK SENSOR** – Replace once a year.



**BATTERY** – Recommended replacement – every 2 years.

### 8.2 Download Log Files

To save log files use a FAT formatted USB mass storage device.

1. Press **Technical** /  and then press **SERVICE**.
2. Enter the code **5844**, and confirm by pressing 

### Download Log Files

3. Press **MORE**.
4. Plug the USB storage device to the ventilator.
5. Change **Auto Logs** to **ON**. The download icon  will display.
6. Wait for the download complete icon  to display. Note that It might take few minutes for download to complete, depending on the size of the log files
7. Check the USB device on a computer and verify that 3 log files – **errors; changes; events** were downloaded.

# 9 Troubleshooting Guide

## 9.1 Introduction

The FLIGHT 60T Ventilator is used in life-support situations. As such, it is essential that all individuals using the FLIGHT 60T Ventilator, including clinicians and support staff, have a thorough understanding of its operation. This should include a working knowledge of the ventilator's pneumatic and electronic systems.

The following practical troubleshooting section is provided as a training resource for individuals learning how to use the FLIGHT 60T Ventilator, and as a reference tool for those already familiar with its use and operation. It should be noted that this outline is not all inclusive, and is intended only as a guide.



Only properly trained personnel should operate the ventilator. The FLIGHT 60T Ventilator is a restricted medical device designed for use by Respiratory Therapists or other properly trained and qualified personnel under the direction of a physician and in accordance with applicable state laws and regulations.

## 9.2 Error Messages

Displayed error	Potential Cause	Suggested Action
CHECK FiO <sub>2</sub> SENSOR	Oxygen sensor bad reading Oxygen sensor defective	Run Oxygen sensor calibration Replace Oxygen sensor
O <sub>2</sub> SUPPLY FAILED	O <sub>2</sub> pressure is very low	Check O <sub>2</sub> source pressure is 30-90PSI Replace the O <sub>2</sub> Regulator filters
PROX LINE	Problem in the proximal line. Pressure sensors is improperly calibrated or defective.	Check internal hoses connections Pressure Sensors Calibration Replace Main Board
FLOW SENSOR ERROR	Processor defective Processor communication problem	Replace Main Board
EMPTY BAT	Combined charge of both batteries is less than 30%.	Connect to AC (charging) for 3 hours Replace Power Board

**Error Messages**

<b>Displayed error</b>	<b>Potential Cause</b>	<b>Suggested Action</b>
HIGH MV INS	Air leakage	Check internal hoses connections
HIGH MV EXH	Calibration problem Hardware problem	Check internal hoses connections Replace Main Board Replace Blower
HIGH / LOW P BASE	Solenoid malfunction	Check solenoid Perform Circuit Test Replace solenoid
PC NOT REACHED	Calibration problem Hardware problem	Replace Blower
LOW MV EXH	Air leak Hardware problem	Check internal hoses connections Replace Main Board
MOTOR FAULT	Motor control Motor	Replace Power Board Replace Blower
PRESS SENSOR	Patient pressure sensor higher than outlet	Perform Pressure sensors calibration Replace Main Board
POWER FAULT	Check log file to identify failed power:	
	3.3V POWER FAULT	Replace Main Board
	MAIN BOARD 10V POWER FAULT	Check cables Replace Power Board Replace Main Board
	POWER BOARD 10V POWER FAULT	Check cables Replace Power Board
	5V POWER FAULT	Replace Main Board
MEMORY FAULT	Parameters mismatch	Perform OVP Replace Main Board
MAIN BAT CHARGER	Charger malfunction	Replace battery Replace Power Board
MAIN BAT V HIGH / LOW	Out of range	Replace battery Replace Power Board

<b>Displayed error</b>	<b>Potential Cause</b>	<b>Suggested Action</b>
MAIN BAT VOLTAGE	Mismatch in Battery voltage reading	Replace battery Replace Main Board
MAIN BAT TEMP HIGH	Battery overheating	Replace battery Replace Power Board
MAIN BAT GAUGE	Battery control	Replace battery
SEC BAT CHARGER	Charger malfunction	Replace battery Replace Power Board
SEC BAT V HIGH / LOW	Out of range	Replace battery Replace Power Board
SEC BAT VOLTAGE	Mismatch in Battery voltage reading	Replace battery Replace Main Board
SEC BAT TEMP HIGH	Battery overheating	Replace battery Replace Power Board
SEC BAT GAUGE	Battery control	Replace battery



## 10 Repackaging and Shipping

All returned items must have a Return Goods Authorization (RGA) number, assigned by FLIGHT MEDICAL.

To obtain an RGA number, email a completed RGA form (filled in its entirety) to [support@flight-medical.com](mailto:support@flight-medical.com). Also include the ventilator's log files and a detailed description of the failure analysis and the repair efforts undertaken.

Items can be returned only after an RGA number was provided by FLIGHT MEDICAL.

Copies of the RGA form and the Service Call should be included with the returned item.

### RGA Returns



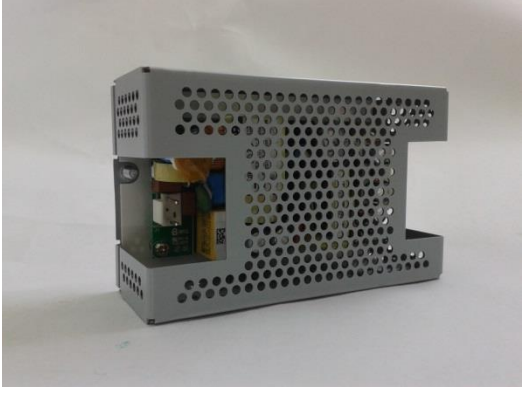
Use the following guidelines to return a FLIGHT 60T ventilator:





- Use the original box and packaging.
- Remove both batteries and any accessories from the ventilator.
- When returning a module, place it in the original box and packaging. If not available, place it in a box large enough to allow for it to be wrapped in a sufficient amount of packaging material.
- Write the RGA number on the shipping label.

Ship RGA items to: FLIGHT MEDICAL INNOVATIONS Ltd.  
7 Hatnufa St,  
Petah Tikva 4951025, ISRAEL  
Tel: +972-3-6731660

# 11 Spare parts

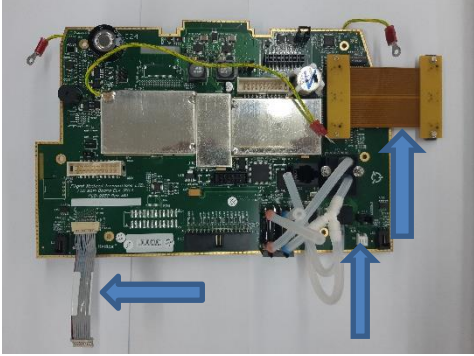


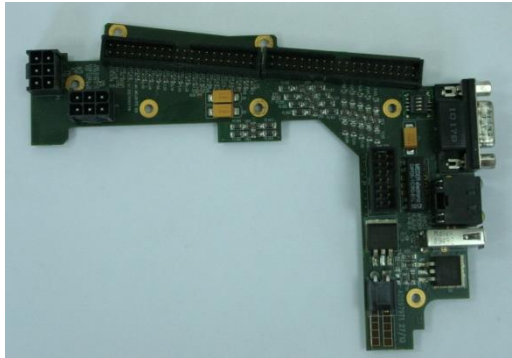

## 11.1 Electronic
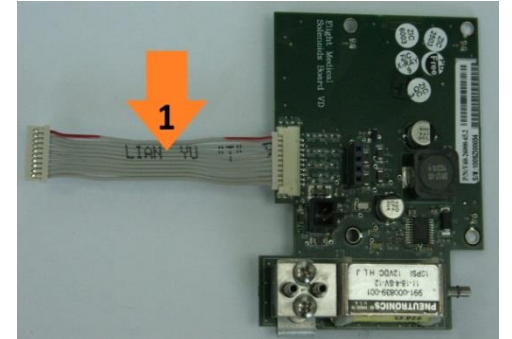
<p style="text-align: center;"><b>Main Board SUB-0198</b></p> <p><i>Connecting cables – not part of the board:</i></p> <table border="1"> <tr> <td style="text-align: center;"><b>1</b></td> <td><b>MB to SB cable</b></td> <td><b>V60-70100-29</b></td> </tr> <tr> <td style="text-align: center;"><b>2</b></td> <td><b>O<sub>2</sub> sensor cable</b></td> <td><b>V60-00130-29</b></td> </tr> <tr> <td style="text-align: center;"><b>3</b></td> <td><b>FPC MB to PB cable</b></td> <td><b>V60-21000-29</b></td> </tr> </table>	<b>1</b>	<b>MB to SB cable</b>	<b>V60-70100-29</b>	<b>2</b>	<b>O<sub>2</sub> sensor cable</b>	<b>V60-00130-29</b>	<b>3</b>	<b>FPC MB to PB cable</b>	<b>V60-21000-29</b>	 <p>The image shows the Main Board SUB-0198, a green printed circuit board (PCB) populated with various electronic components. Three blue arrows point to specific connection points: one on the left edge, one on the right edge, and one near the bottom center. A white cable is also visible connected to the board.</p>
<b>1</b>	<b>MB to SB cable</b>	<b>V60-70100-29</b>								
<b>2</b>	<b>O<sub>2</sub> sensor cable</b>	<b>V60-00130-29</b>								
<b>3</b>	<b>FPC MB to PB cable</b>	<b>V60-21000-29</b>								
<p style="text-align: center;"><b>Power Board ELA-0020</b></p>	 <p>The image shows the Power Board ELA-0020, a green PCB with several integrated circuits, capacitors, and other electronic components. It has a complex, irregular shape with multiple connection points.</p>									
<p style="text-align: center;"><b>Power Supply SUB-0174</b></p>	 <p>The image shows the Power Supply SUB-0174, a silver metal enclosure with a perforated front panel. The internal components, including a power supply unit and other electronics, are visible through the open side of the enclosure.</p>									





<p><b>Lower Board</b> V60-22000-65</p>				
<p><b>D-Type Board</b> V60-24000-65</p>				
<p><b>Purge Board</b> V60-23000-65</p>				
<p><b>Solenoid Board</b> V60-26000-65</p> <p><i>Connecting cables – not part of the board:</i></p> <table border="1" data-bbox="261 1562 839 1612"> <tr> <td>1</td> <td>MB to SB cable</td> <td>V60-70100-29</td> </tr> </table>	1	MB to SB cable	V60-70100-29	
1	MB to SB cable	V60-70100-29		

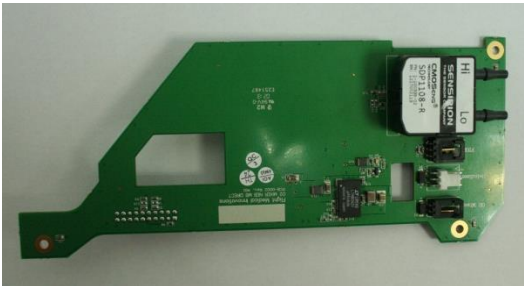Spare parts

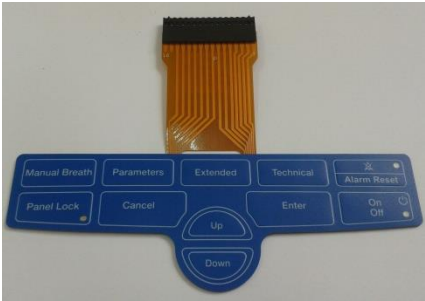Electronic



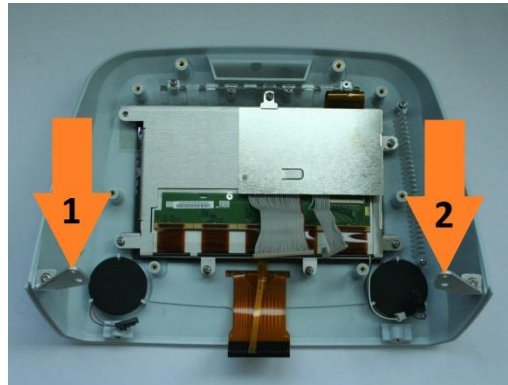

<p><b>O<sub>2</sub> Mixer Neb board</b> <b>ELE - 0001</b></p>	 A green printed circuit board (PCB) for an O2 Mixer Neb. It features a Siemens SIMATIC 3ES7 138-6EX00-0AB0 power supply module on the right side. The board has several electronic components, including resistors and integrated circuits, and is populated with gold-plated pins along its edges.
<p><b>Detachable (main) battery</b> <b>V60-19000-67</b></p>	 A blue, rectangular, detachable main battery. It has a prominent blue latch on the left side with the text "OPEN" and "REMOVABLE BATTERY" visible. The battery is shown from a three-quarter perspective.
<p><b>Integral battery</b> <b>SUB-0222</b></p>	 A blue, rectangular integral battery. The top surface is printed with technical specifications in multiple languages (English, German, French) and the part number "P/N SUB-0222". It has a textured surface and a small latch on the right side.
<p><b>Keypad - English</b> <b>V60-74000-68</b></p>	 A blue keypad with an orange ribbon cable attached to the top. The keypad features several function buttons labeled: "Manual Breath", "Parameters", "Extended", "Technical", "Alarm Reset", "Panel Lock", "Cancel", "Enter", "On/Off", "Up", and "Down".
<p><b>Keypad - Symbols</b> <b>SUB-0087</b></p>	 A blue keypad with an orange ribbon cable attached to the top. The keypad features several function buttons with symbols: a person icon, a house icon, a gear icon, a wrench icon, a power icon, a lock icon, a plus icon, an "OK" button, and a minus icon.




## 11.2 Mechanical / Assemblies

<p style="text-align: center;"><b>Front Subassembly</b></p> <p style="text-align: center;">SUB-0195 - English Keypad SUB-0194 - Symbols Keypad</p> <p><i>Connected items:</i></p> <table border="1"> <tbody> <tr> <td style="text-align: center;">1</td> <td><b>Front right bracket</b></td> <td><b>V60-76023-03</b></td> </tr> <tr> <td style="text-align: center;">2</td> <td><b>Front Left bracket</b></td> <td><b>V60-76022-03</b></td> </tr> </tbody> </table>		1	<b>Front right bracket</b>	<b>V60-76023-03</b>	2	<b>Front Left bracket</b>	<b>V60-76022-03</b>	 <p>A photograph of the front subassembly assembly. Two orange arrows labeled '1' and '2' point to the front right and left brackets respectively, which are being attached to the main keypad assembly.</p>
1	<b>Front right bracket</b>	<b>V60-76023-03</b>						
2	<b>Front Left bracket</b>	<b>V60-76022-03</b>						
<p style="text-align: center;"><b>Blower</b></p> <p style="text-align: center;">SUB-0191</p> <p><i>Connected item:</i></p> <table border="1"> <tbody> <tr> <td style="text-align: center;">1</td> <td><b>Cooling Fan assy.</b></td> <td><b>MEB-0038</b></td> </tr> </tbody> </table>		1	<b>Cooling Fan assy.</b>	<b>MEB-0038</b>	 <p>A photograph of the blower assembly, showing a cooling fan mounted on a metal base with various electrical connections and wires.</p>			
1	<b>Cooling Fan assy.</b>	<b>MEB-0038</b>						
<p style="text-align: center;"><b>Solenoid</b></p> <p style="text-align: center;">SUB-0280</p>		 <p>A photograph of the solenoid assembly, a cylindrical component with a green cap and a blue connector, with electrical wires attached.</p>						
<table border="1"> <tbody> <tr> <td style="text-align: center;">1</td> <td><b>Filter Cover</b></td> <td><b>V11-35201-07</b></td> </tr> <tr> <td style="text-align: center;">2</td> <td><b>O-ring</b></td> <td><b>V11-36000-26</b></td> </tr> </tbody> </table>		1	<b>Filter Cover</b>	<b>V11-35201-07</b>	2	<b>O-ring</b>	<b>V11-36000-26</b>	 <p>A photograph showing the filter cover and O-ring. The filter cover is a circular component with a handle and a red O-ring. Two orange arrows labeled '1' and '2' point to the filter cover and O-ring respectively.</p>
1	<b>Filter Cover</b>	<b>V11-35201-07</b>						
2	<b>O-ring</b>	<b>V11-36000-26</b>						



Spare parts

Mechanical / Assemblies

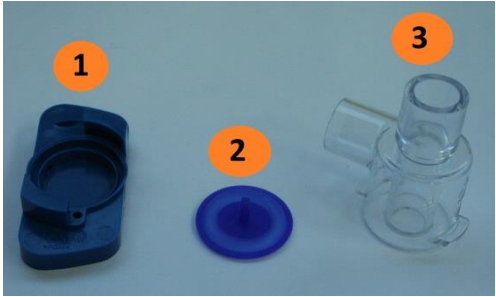


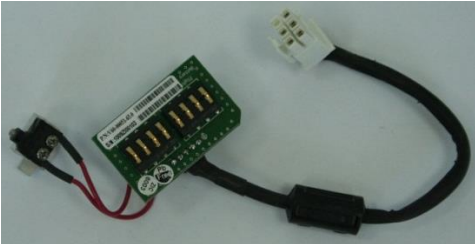


<p><b>Oxygen Sensor / Leak sensor</b> <b>G60-25000-29</b></p>	 <p>A white cylindrical oxygen sensor with a black label. The label contains the following text: "Medical Instrumentation", "P.O. Box 9712", "Pomona, CA 91768", "info@mh1.com", "MHT OXYGEN SENSOR", "SN: 100-9712-01", and "Caustic Material SN: 100-9712-01".</p>
<p><b>Inlet Assembly</b> <b>SUB-0182</b></p>	 <p>A clear plastic inlet assembly with a circular filter cover. The cover has the following text: "FILTER COVER", "LOOSEN SCREWS TO REMOVE COVER. FILTER IS TO BE CHECKED WEEKLY AND REPLACED WHEN DIRTY. DO NOT REVERSE FILTER WHEN DIRTY.", "ATTACHMENT SOCKET FOR OXYGEN BLENDER", "FRESH GAS INTAKE", "DO NOT BLOCK!", "SECURELY TIGHTEN SCREWS!".</p>
<p><b>Internal Mixer Nebulizer</b> <b>SUB-0179</b></p>	 <p>An internal mixer nebulizer assembly consisting of a clear plastic chamber, a metal mixing tube, and a blue solenoid valve. A black cable is connected to the solenoid valve.</p>
<p><b>Cooling Fan</b> <b>MEB-0038</b></p>	 <p>A black cooling fan with a white label. The label contains the following text: "3600 RPM", "12VDC", "0.15A", "1.8W", "1.5\" diameter", and "1.5\" height".</p>

<p style="text-align: center;"><b>Inlet Blower</b></p> <p style="text-align: center;"><b>MEC-0140</b></p>		
<p style="text-align: center;"><b>Outlet Assembly Blower</b></p> <p style="text-align: center;"><b>SUB-0183</b></p>		
<p><b>For Dual Limb Ventilator</b></p>		
1	<p><b>Dual limb Cap</b></p> <p style="text-align: right;"><b>SUB-0088</b></p>	
2	<p><b>Diaphragm (5 pcs)</b></p> <p style="text-align: right;"><b>V64-00050-90</b></p>	
3	<p><b>Exhalation Valve DL</b></p> <p style="text-align: right;"><b>MEC-0034</b></p>	

### 11.3 Cables


<p style="text-align: center;"><b>Internal AC Cable</b></p> <p style="text-align: center;"><b>CBL-0078</b></p>	
<p style="text-align: center;"><b>Internal DC Cable</b></p> <p style="text-align: center;"><b>V60-40200-29</b></p>	

Cables

<p>FPC MB-PB cable V60-21000-29</p>	
<p>MB – SB cable V60-70100-29</p>	
<p>Detachable Battery cable V60-00050-29</p>	
<p>Internal Battery cable V60-50100-29</p>	
<p>Oxygen Sensor cable V60-00130-29</p>	
<p>O<sub>2</sub> Leak sensor cable SUB-0010</p>	

DC-DC cable (power supply – Power board)

CBL-0054

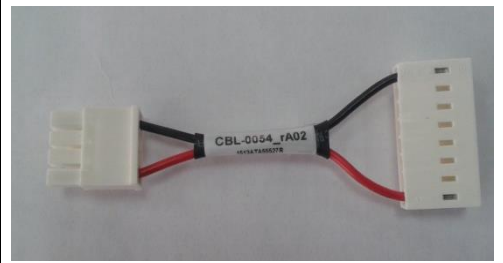



# Index

## A

- AC connector, 3-6
- ACMV, 6-3, 7-12, 7-18
- Air/Oxygen Entrainment Mixer, 2-2, 2-3, 3-5, 7-10
- Alarm connector, 3-6
- Alarm Reset button, 3-1, 3-3
- Antistatic wrist strap, 4-1
- APNEA, 3-3
  - LED Indicator, 3-3
  - LED Indicator, 7-12
- Audio Paused button, 3-1
- Audio Paused/Alarm Reset
  - LED Indicator, 7-13

## B

- Back panel
  - AC connector, 3-6
  - DC connector, 3-6
  - Detachable battery, 3-6
  - Low flow oxygen port, 3-7
  - Network logging, 3-7
  - PC connector, 3-6
  - Peripherals connector, 3-7
  - Remote alarm connector, 3-6
  - Uploading log files, 3-7
- BAT
  - LED Indicator, 3-3, 7-19, 7-20, 9-3, 9-4, 9-5
  - LED Indicator (Battery power), 7-12
- Battery, 4-2, 7-19, 9-5
- Button
  - Alarm Reset, 3-1, 3-3
  - Audio Paused, 3-1
  - Cancel, 3-3
  - Decrease, 3-3
  - Extended, 3-3
  - Home, 3-2
  - Increase, 3-3
  - Manual breath, 3-1
  - Manual breath, 3-2
  - OK, 3-3
  - On/Off, 3-1, 3-3
  - Panel lock, 3-1
  - Panel lock, 3-3
  - Parameters, 3-2
  - Technical, 3-3
  - Up/Down, 3-1
- BUV
  - LED Indicator, 3-3
  - LED Indicator (Back up ventilation), 7-12
- Buzzer Test, 7-20

## C

- Cable ties, 4-1
- Cancel button, 3-3
- Charger, 7-19, 9-4, 9-5
- Connector, patient circuit, 3-1
- Ventilator Cover, 4-3, 4-5, 4-6
- Cutter, 4-1

## D

- DC connector, 3-6
- Decrease button, 3-3
- Detachable battery, 3-6
- Detachable Battery, 4-2
- Display touch screen, 3-2
- D-type Board, 4-26
- Dual limb exhalation valve, 3-2

## E

- Electrical Diagram, 3-11
- Emergency Air Intake, 3-4
- Error Messages, 9-3
- Error! Reference source not found. Calibration, 7-13
- EXT PWR
  - LED Indicator, 3-3
  - LED Indicator (External power), 7-12
- Extended button, 3-3

## F

- FAULT
  - LED Indicator, 3-3, 9-4
  - LED Indicator (Device malfunction), 7-12
- FILTER, 8-1**
- Filter Cover, 3-5
- Flat screwdriver, 4-1, 7-5
- Fresh Gas Intake, 3-5
- Front Panel, 3-1, 4-10, 4-11, 4-12, 4-15, 7-12
- Front Panel Assembly, 4-10, 4-12
- Front Panel Self Test, 7-12
- Fuse, 4-28

## H

- HIGH MV
  - LED Indicator, 3-3, 9-4
  - LED Indicator (High minute ventilation), 7-12
- High P
  - LED Indicator (High pressure), 7-13
- HIGH P
  - LED Indicator, 3-3, 7-18, 7-19

High Pressure Alarm, 7-18  
Home button, 3-2

### I

Increase button, 3-3  
Internal Battery, 4-2, 4-3

### L

LED indicator, 3-2  
LED Indicators, 3-3  
LOW MV  
    LED Indicator (Low minute ventilation), 7-13  
LOW BAT  
    LED Indicator, 3-3  
    LED Indicator (Low battery), 7-12  
Low flow oxygen port, 3-7  
LOW MV  
    LED Indicator, 3-3, 9-4  
LOW P  
    LED Indicator, 3-3, 9-4  
    LED Indicator (Low pressure), 7-13  
Lower Board, 4-26

### M

Main Board, 4-10, 4-13, 4-15, 4-16, 7-13, 7-19, 7-20, 7-21, 9-3, 9-4, 9-5  
Maintenance, 8-1  
MANIFOLD, 8-1  
Manifold Assembly, 4-17, 4-18  
Manual breath button, 3-1, 3-2

### N

Network logging, 3-7

### O

OK button, 3-3  
On/Off, 7-12  
    LED Indicator, 7-13  
On/Off button, 3-1, 3-3, 7-12  
Oxygen Blending Bag Kit, 2-2, 2-3  
Oxygen Sensor, 4-18, 4-19, 7-20

### P

Panel Lock  
    LED Indicator, 7-13  
Panel lock button, 3-1, 3-3  
Parameters button, 3-2  
Patient Circuit  
    Reusable, 7-5

Patient circuit connector, 3-1  
PC connector, 3-6  
PEEP, 6-3, 7-12, 7-15, 7-16  
Peripherals connector, 3-7  
Phillips screwdriver PH1, 4-1, 7-5  
PLT1M-M69, 4-1  
PLT4S-M69, 4-1  
Power Board, 4-6, 4-7, 4-8, 9-3, 9-4, 9-5  
Power Supply, 4-9, 4-10, 7-19  
Pressure gauge, 3-2  
Pressure Relief Calibration, 7-8  
Pressure Sensors Calibration, 7-5, 9-3  
Pressure Trigger, 7-17  
Pressure Verification, 7-14  
Primary alarm LED, 3-2  
Purge Board Assembly, 4-22

### R

Remote alarm connector, 3-6  
Removing and Reinstalling Modules, 4-1  
Repackaging, 10-1  
RGA number, 10-1  
Rubber Bumpers, 4-29

### S

**OXYGEN SENSOR**  
    **Oxygen, 8-1**  
    Shipping, 10-1  
    Solenoid Assembly, 4-25  
    Solenoid Board Assembly, 4-23, 4-24  
    Standard Ventilator Settings (STS), 7-12, 7-14, 7-16, 7-17, 7-18

### T

Technical button, 3-3  
Test Table, 7-25  
Torx screwdriver BT10x200, 4-1, 7-5  
Touch Screen, 4-16  
TRIG  
    LED Indicator, 3-3  
    LED Indicator (Pressure trigger), 7-12  
Troubleshooting Guide, 9-3

### U

Uploading log files, 3-7

### V

Verification, 2-3, 7-5  
Volume Factor Calibration, 7-9