


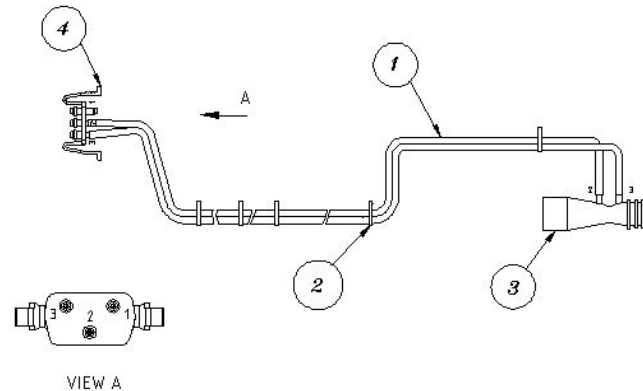

FLIGHT MEDICAL INNOVATIONS Ltd.

Flight60 Dual Limb Pediatric Single Use Flow Sensor Kit (1.8m)

REF V64-0024


INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to a **pediatric** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

1. Flow Transducer Lines
2. Tube Clamps
3. Flow Orifice
4. Quick Connector

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach a breathing circuit (authorized by Flight Medical) through all evenly spread Tube Clamps (2). Avoid tubes kinking.
2. Connect the proximal end of the breathing circuit to the distal end of Flow Orifice (3).
3. Prior to connecting the patient, perform a circuit test according to the instructions in the F60 Operator's Manual.
4. Store in a cool and dry place.
5. Discard circuit after a single use.

CLEANING/SANITAZATION CONDITIONS:

Autoclave Temp.: 134°C; **Time:** 4 min; **Drying Time:** 15 min.

DO NOT USE THE FOLLOWING FOR CLEANING: Phenol (>5%), Ketones, Formaldehyde, Chlorinated/Aromatic Hydrocarbons, Inorganic Acid. Do not Autoclave if medications containing Quaternary Ammonium, Chlorinated or Aromatic Hydrocarbons have been used.

WARNINGS:

This circuit is designed for SINGLE PATIENT USE ONLY. DO NOT STERILIZE

This product is to be assembled only into a Dual Limb breathing circuit authorized by Flight Medical.

This product is designed to be used only with F60 DL Ventilators.

The use of a Dead Space Reducer should be considered when ventilating pediatric patients.

Federal (U.S.A) law restricts this device for sale by or on the order of a physician.

