


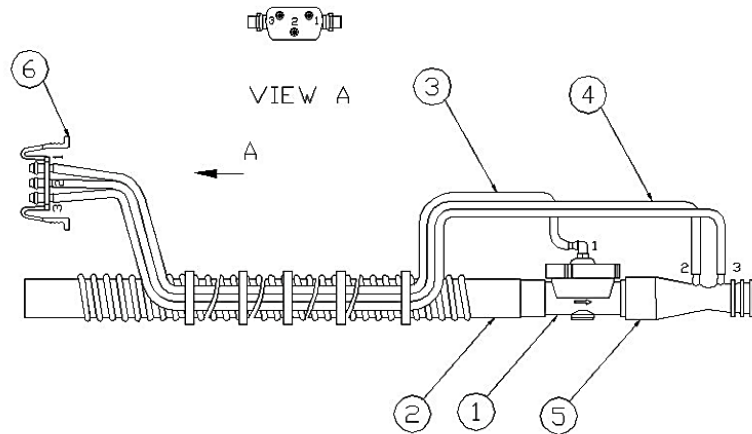

FLIGHT MEDICAL INNOVATIONS Ltd.

Flight60 Single Limb Ped. Single Use Patient Circuit w/RD (1.8m)

REF V64-0069


INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to a **pediatric** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

- | | |
|--------------------------|------------------------------|
| 1. Exhalation Valve | 4. Flow Transducer Lines |
| 2. Inspiratory Limb | 5. Flow Orifice with reducer |
| 3. Exhalation Drive Line | 6. Quick Connector |

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach the Quick Connector (6) to its port in the front panel and secure tightly.
2. Attach the Inspiratory Limb (2) to the Gas Outlet in the front panel.
3. If using with an HMEF, attach the HMEF to the Flow Orifice (5).
4. Prior to connecting the patient, perform a circuit test according to the instructions in the F60 Operator's Manual.
5. Store in a cool dry place.
6. Discard circuit after a single use.

TECHNICAL SPECIFICATION:

R@ 30 l/min: 0.006 (cmH₂O/l/min).

C@ 60 cmH₂O: 1.16 (ml/cmH₂O).

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE.

This circuit is designed to be used only with F60 Ventilators.

The use of a Dead Space Reducer should be considered when ventilating pediatric patients.

Federal (U.S.A) law restricts this device for sale by or on the order of a physician.