

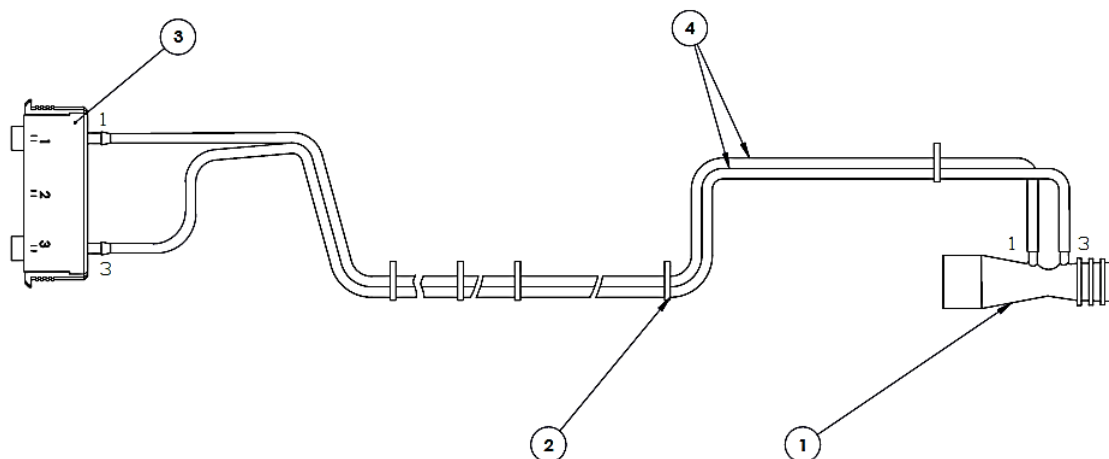
Obelis s.a

Address: Boulevard Général Wahis 53, 1030 Brussels, Belgium

TEL: +32-2-7325954, FAX: +32-2-732-6003

FLIGHT MEDICAL INNOVATIONS Ltd.

VX-DL Adult Single Use F.S. Kit (1.8m)

REF VX64-0005


INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to an **adult** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

Ventoux - Dual Limb Adult Single Use Flow Sensor Kit

1. Flow Orifice
2. Tube Clamps
3. Quick Connector
4. Flow Transducer Lines

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach a breathing circuit (authorized by Flight Medical) through all evenly spread Tube clamps (2). Avoid tubes kinking.
2. Connect the proximal end of the breathing circuit to the distal end of Flow Orifice (1).
3. Prior to connecting the patient, perform a circuit test according to the instructions in the Ventoux Operator's Manual.
4. Store in a cool and dry place.
5. Discard circuit after a single use.

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE

This product is to be assembled only into a Dual Limb breathing circuit authorized by Flight Medical.

This product is designed to be used with Ventoux Ventilators only.

Federal (U.S.A) law restricts this device for sale by or on the order of a physician.