


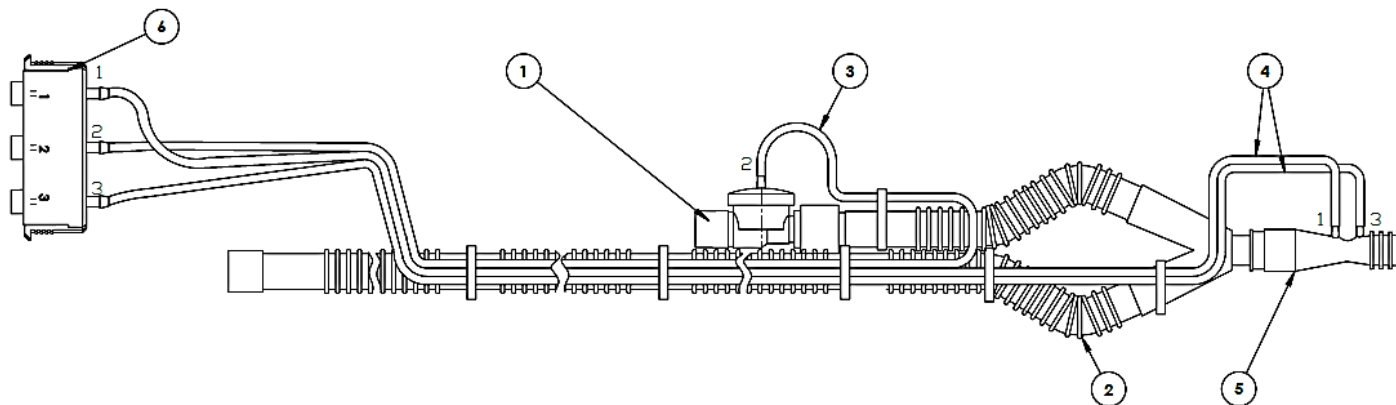

FLIGHT MEDICAL INNOVATIONS Ltd.

VX-SL Ped. Single Use P.C. (1.8m)

REF VX64-0013


INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to a **pediatric** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

Ventoux – Single Limb Pediatric Single Use Patient Circuit

- | | |
|--------------------------|--------------------------|
| 1. Exhalation Valve | 4. Flow Transducer Lines |
| 2. Inspiratory Limb | 5. Flow Orifice |
| 3. Exhalation Drive Line | 6. Quick Connector |

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach the Inspiratory Limb (2) to the Gas Outlet in the front panel.
2. Attach the Quick Connector (6) to its port in the front panel and secure tightly.
3. If using an HMEF, attach the HMEF to the Flow Orifice (5).
4. Prior to connecting the patient, perform a circuit test according to the instructions in the Ventoux Operator's Manual.
5. Store in a cool and dry place.
6. Discard circuit after a single use.

TECHNICAL SPECIFICATION:

R@ 30 l/min: 0.006 (cmH₂O/l/min).

C@ 60 cmH₂O: 1.16 (ml/cmH₂O).

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE.
 This product is designed to be used only with Ventoux Ventilators.
 The use of a Dead Space Reducer should be considered when ventilating pediatric patients.
 Federal (U.S.A) law restricts this device for sale by or on the order of a physician.