


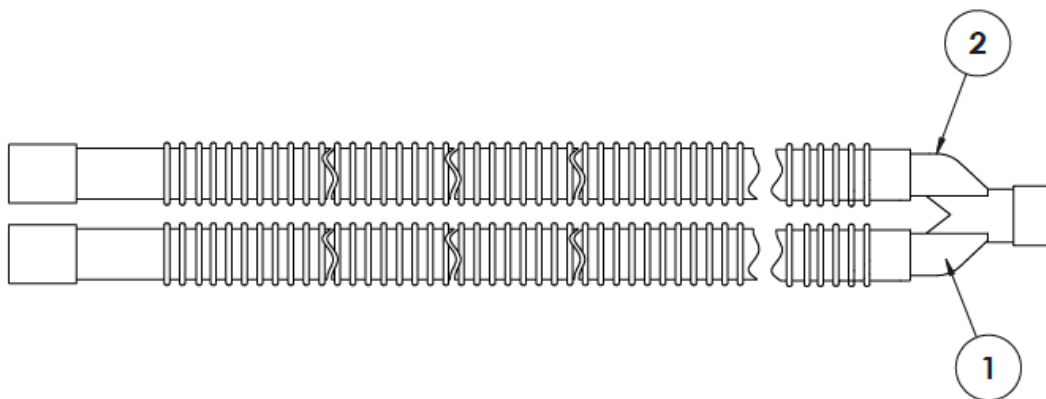

FLIGHT MEDICAL INNOVATIONS Ltd.

VX-DL Adult Single Use P.C. Universal (1.8m)

REF VX64-0001


INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to an **adult** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

Ventoux - Dual Limb Adult Single Use Patient Circuit Universal

1. Inspiratory Limb
2. Expiratory Limb

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach the Inspiratory Limb (1) to the Gas Outlet in the front panel.
2. Attach the Expiratory Limb (2) to the Exhalation Valve in the front panel.
3. Prior to connecting the patient, perform a circuit test according to the instructions in the Ventoux Operator's Manual.
4. Store in a cool and dry place.
5. Discard circuit after a single use.

TECHNICAL SPECIFICATION:

R@ 30 l/min: 0.006 (cmH₂O/l/min).

C@ 60 cmH₂O: 1.5 (ml/cmH₂O).

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE.
This circuit is designed to be used only with Ventoux Ventilators.
Federal (U.S.A) law restricts this device for sale by or on the order of a physician.