


FLIGHT MEDICAL INNOVATIONS Ltd.

VX-SL Ped. Single Use P.C. w/RD

REF VX64-0019

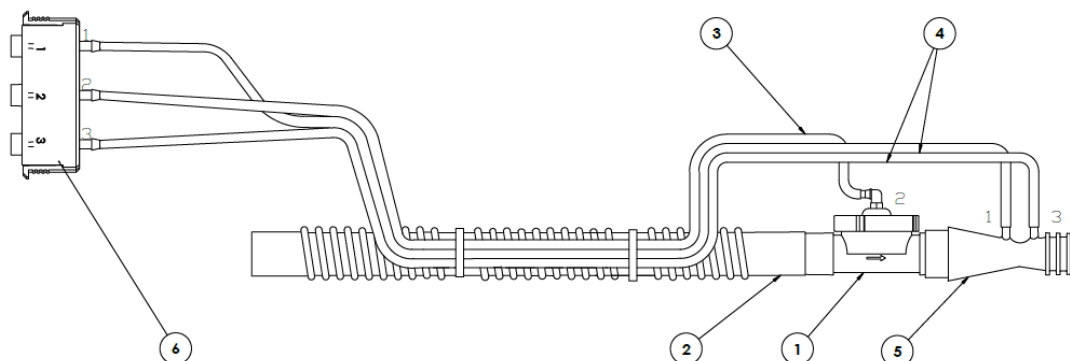

INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to a **pediatric** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

Ventoux - Single Limb Pediatric Single Use Patient Circuit with Reducer

1. Exhalation Valve
2. Inspiratory Limb
3. Exhalation Drive Line
4. Flow Transducer Lines
5. Flow Orifice
6. Quick Connector

INSTRUCTIONS FOR USE:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.
4. Attach the Inspiratory Limb (2) to the Gas Output on the front panel.
5. Attach the Quick Connector (1) to its socket on the front panel and secure tightly.
6. If using an HME, attach the HME to the Flow Orifice (3).
7. Prior to connecting the patient, perform a circuit test according to the instructions in the Ventoux Operator's Manual.
8. Store in a cool dry place.

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE.
 This circuit is designed to be used only with Ventoux Ventilators.
 The use of a Dead Space Reducer should be considered when ventilating pediatric patients.
 Federal (U.S.A) law restricts this device for sale by or on the order of a physician.