


FLIGHT MEDICAL INNOVATIONS Ltd.

Flight60 Single Limb Single Use Flow Sensor Kit (1.8m)

REF V64-50120-90

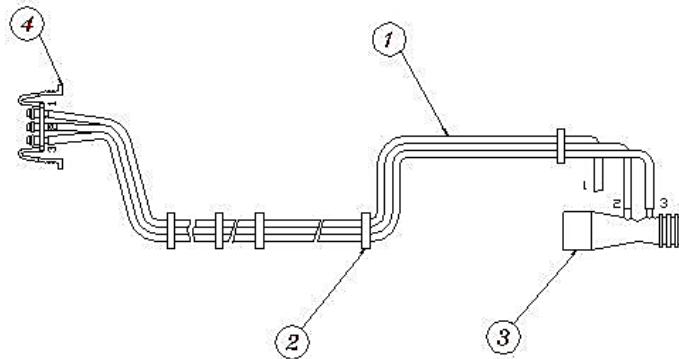

INTENDED USE:

The breathing circuit is intended to administer gases such as oxygen, air or a combination to an **adult** patient requiring Ventilation. It is a part of the interface between a mechanical ventilator and the patient for inhalation and exhalation.

CONTENT:

1. Exhalation Drive Line
2. Tube Clamps
3. Flow Orifice
4. Quick Connector

CAUTION:

1. The content of the bag is clean but not sterile.
2. Open the packaging with care. Avoid using sharp objects as they may harm the product.
3. Inspect the product prior to use to ensure that all components are intact.

INSTRUCTIONS FOR USE:

1. Attach a breathing circuit (authorized by Flight Medical) through all evenly spread tube clamps (2). Avoid tubes kinking.
2. Connect the Exhalation Drive Line (1) to exhalation valve's drive port (authorized by Flight Medical).
3. Connect the proximal end of the exhalation valve to the distal end of Flow Orifice (3).
4. Prior to connecting the patient, perform a circuit test according to the instructions in the F60 Operator's Manual.
5. Store in a cool and dry place.
6. Discard circuit after a single use.

WARNINGS:

This circuit is designed for SINGLE USE ONLY. DO NOT STERILIZE.

This product is to be assembled into a breathing circuit and an exhalation valve authorized by Flight Medical only.

This product is designed to be used with F60 Ventilators only.

Federal (U.S.A) law restricts this device for sale by or on the order of a physician.