



On Screen Trends

Valuable patient history at your fingertips

Flight 60® Ventilators now display up to 72 hours trends.

Total Rate, Peak Inspiratory Pressure (PIP) and Exhaled Tidal Volume (Vte) trends are presented on the Flight 60 screen, featuring quick access and auto scaling for immediate presentation and analysis.





Trend Settings

To activate Trends view, press the "Extended" button, tap "Trends" and choose "ON".

Trends are displayed on the graphics screen area; tap the "Loops" display screen in order to view the trended parameters.



To clear the trends memory, press the "Extended" button, tap "Trends" and choose "CLEAR".

Selecting Trended Parameters

The following parameters can be trended:

1. Actual Rate
2. Peak Inspiratory Pressure (PIP)
3. Exhaled Volume (Vte)



Tap the monitoring area to toggle between the trend displays: Rate, PIP and Vte.

Time Scale Adjustment

To change the time interval of the trended period, tap the left and right arrows « » located at the upper corners of the trend display.

